

Quick Start Guide



Get To Know The Compact Go

Front Side:




Rear Side



Menu Button

Power Button

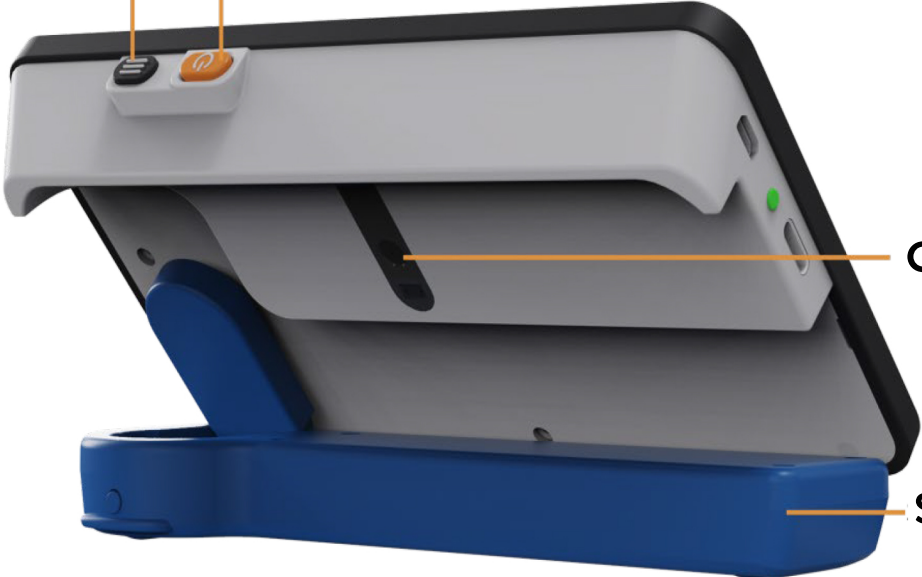

Cameras

Stand

Wireless Charge Cradle



Charging Port

2. Charging

Wired charging: Using the provided adapter and USB cable, connect the USB cable to the Compact Go's USB-C port to charge.

Wireless charging: Connect the wireless charging cradle to the power supply via the charging port and the provided USB-C cable. Then, put Compact Go on the cradle in the matching orientation.

3. Open and Use

Power on/off: Long press the orange power button for 2s or open the stand to turn on the Compact Go. Press the power button for 2s to turn off the unit. Switching on and off is accompanied by a beeping buzzer.

Different view modes: Open the stand for near view; close the stand and rotate it as a handle for distance view as below.



Contact Details:



Optelec US
17757 US Hwy 19 N, Suite 200
Clearwater, FL 33764
U.S.A.
T: +1 (800) 444-4443
www.optelec.com