Magnification

Compact 10 HD **Compact 10 HD Speech**

Advanced Features and Functions

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Optelec

Contents

ADVANCED FEATURES AND FUNCTIONS	17
1. INTRODUCTION	17
1.1. ABOUT THIS MANUAL	18
1.2. WHAT'S IN THE BOX?	19
1.3. WHAT'S ON THE SCREEN?	20
2. GETTING STARTED	25
2.1. CHARGING THE BATTERY	25
<i>Battery warning</i>	26
2.2. POWER ON AND POWER OFF	26
<i>Standby</i>	27
2.3. FOLDING AND UNFOLDING THE COMPACT 10 HD	27
2.4. CLEANING THE CAMERAS	29
2.5. BUTTONS AND CONNECTIONS	30
3. CAMERAS	31
3.1. MAGNIFIER CAMERA	31
3.2. FULL PAGE CAMERA	32
<i>Focus lock</i>	32
3.3. OVERVIEW CAMERA	32
4. USING TEXT-TO-SPEECH & OCR	33
4.1. ACTIVATING THE SPEECH	33
4.2. ADJUSTING THE VOLUME AND READING SPEED	34

4.3. ADJUSTING LANGUAGE AND VOICE.....	34
4.4 TABLE MODE.....	35
5. EASY AND ADVANCED MODES	36
5.1 FEATURES IN EASY MODE	38
<i>Adjusting the magnification</i>	38
<i>Selecting a viewing mode</i>	38
<i>Taking a snapshot of an image (Compact 10 HD only)</i>	39
<i>Taking a snapshot and use Text-to-Speech (Compact 10 HD Speech)</i>	39
<i>Contrast Adjustment</i>	39
5.2. FEATURES IN ADVANCED MODE	40
<i>Time and date</i>	40
<i>Taking a snapshot of an image (Compact 10 HD Speech)</i>	40
5.3 MENU	41
<i>Opening and closing the menu</i>	41
<i>Voice (Compact 10 HD Speech only)</i>	43
<i>Line On/Off</i>	44
<i>Battery status</i>	44
<i>Save</i>	44
<i>Open</i>	45
<i>Delete</i>	45
<i>Adjusting the brightness</i>	45
5.4. SETTINGS.....	46
<i>Color</i>	47
<i>View (Compact 10 HD Speech only)</i>	48
<i>Miracast: connecting wirelessly to a monitor or TV</i>	48
<i>Bluetooth (Compact 10 HD Speech only)</i>	49
<i>Line</i>	50

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1. Introduction

Congratulations on choosing the Compact 10 HD or the Compact 10 HD Speech!

With its large 10" screen, customizable features and intuitive operation, the Compact 10 HD offers all benefits of a desktop video magnifier, but in a much smaller, foldable, and portable design. With the arm unfolded, you can easily look at photos, write your signature, view medicine packaging, and even read round cans without being restricted in your movements. With the Compact 10 HD Speech, the swing-out arm allows you to scan and read full pages of text.

If you have questions or suggestions concerning the use of this product, contact your distributor or Optelec's headquarters, using the contact information on the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your Compact 10 HD!

1.1. About this manual

At Optelec, we are constantly improving our products and their functionality. Therefore, it is possible that this manual is not the latest version. Download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and basic operation of the Compact 10 HD and the Compact 10 HD Speech. Read this manual carefully before using your device.

Unless otherwise indicated, all references to the Compact 10 HD will also apply for the Compact 10 HD Speech. Functionality specifically applicable for the Compact 10 HD Speech are marked with the Speech logo.

1.2. What's in the box?

The Compact 10 HD packaging contains the following:

- The Compact 10 HD or the Compact 10 HD Speech video magnifier
- A protective pouch
- A USB power supply with USB-C cable
- 4 country-specific power plugs (EU, US, UK, and AU)
- A screen cleaning cloth
- Setup and Use
- Safety and Care

If any of these items are missing from your package, contact your distributor.

1.3. What's on the screen?

Icons are used on the Compact 10 touch screen to represent product functionality. Use this section as a reference tool for understanding the graphics on the screen.

Some graphics only appear if you own a Compact 10 Speech. The table below specifies which functions are available in Speech only. In addition, a blue icon in the documentation indicates Speech only.

Icon	Description	Both or Speech only	Occurrence
	Decrease the magnification of the image on the screen.	Both	Easy and Advanced modes
	Take a snapshot of an image without using text to speech.	Both	Easy and Advanced modes
	Take a snapshot, scan for text, and read the text.	Speech only	Advanced mode
	Change the combination of colors used to display the live view.	Both	Easy and Advanced modes

Icon	Description	Both or Speech only	Occurrence
	Increase the magnification of the image on the screen.	Both	Easy and Advanced modes
	Open the main menu, where you can change system settings.	Both	Advanced mode
	Displays the current time and date.	Both	Advanced mode
	Set focus lock so you can maintain the focus on one item while another is moved in the camera frame.	Both	Full Page camera in Easy and Advanced modes
	Tap to open the volume and reading speed menu. Long press to select a voice.	Speech only	After taking a Text-to-Speech snapshot in Easy and Advanced modes

Icon	Description	Both or Speech only	Occurrence
	Close the snapshot.	Both	After taking a snapshot in Easy and Advanced modes
	Adjust the volume.	Speech only	After opening the volume and reading speech menu in Easy and Advanced modes
	Adjust reading speed.	Speech only	After opening the volume and reading speech menu in Easy and Advanced modes
	Play or pause reading.	Speech only	After text to speech

Icon	Description	Both or Speech only	Occurrence
	Tap to change the viewing mode. Long press to switch between Basic and Advanced modes.	Both	Easy and Advanced modes
	Add, remove, and select your reading voice and language and your menu voice and language.	Speech only	When changing menu settings in Advanced mode
	Determine whether highlighting, underlining, or boxes are used to show the current place in captured text.	Speech only	When changing menu settings in Advanced mode
	Use Miracast to connect to a television or monitor via Wi-Fi.	Both	When changing menu settings in Advanced mode
	Use Bluetooth to pair with a speaker or headphones via Wi-Fi.	Speech only	When changing menu settings in Advanced mode

Icon	Description	Both or Speech only	Occurrence
	Choose whether to display a reading line across the screen.	Both	When changing menu settings in Advanced mode
	Change the position of the button bar.	Both	When changing menu settings in Advanced mode
	Shows the status of the battery. A lightning symbol means the battery is charging.	Both	From main menu in Advanced mode
	Brightness slider.	Both	From main menu in Advanced mode
	Display a menu where you change settings, including the way images look on the screen and how sound is handled.	Both	From main menu in Advanced mode

2. Getting Started

The Compact 10 HD is placed correctly in front of you when the screen is facing up and towards you, with the Optelec logo below the screen.

2.1. Charging the battery

The Compact 10 HD is delivered with a power supply and four country-specific power plugs in EU, US, UK, and AU types. Slide the appropriate plug for your region onto the power supply until it clicks.

The battery should be charged before initial use. To charge the Compact 10 HD, connect the supplied USB cable to the power supply and the blue USB connector located on the left side of the unit. Plug the other end of the power supply into a power outlet.

Battery full

Battery charging

Battery almost empty

Charging the Compact 10 HD takes about 3.5 hours. A fully charged Compact 10 HD has up to 3.5 hours of battery life, depending on product settings and use. For safety reasons, only use the power supply and USB cable that was delivered with the Compact 10 HD to charge the unit's battery.

Battery warning

The Compact 10 HD will briefly show an empty battery warning icon to indicate that the battery needs to be charged. When this occurs, make sure to charge your unit. When the battery is empty, the unit will automatically switch off.

If the Compact 10 HD does not switch on, make sure it is charged.

2.2. Power on and power off

To switch on the Compact 10 HD for the first time, the unit needs to be charged. Once charged, hold down the orange On / Off switch for three seconds. To switch off the Compact 10 HD, press the orange button for three seconds again. When switching on the Compact 10 HD, it will take approximately 20 seconds before an image displays on screen.

Standby

Press the orange On / Off button briefly to put Compact 10 HD in standby mode. Compact 10 automatically goes into standby mode if no button is pressed and no movement is detected for a user-defined amount of time. Briefly pressing the On / Off button powers it back on. You can change or disable the standby time in the Compact 10 HD menu. See chapter 5.4 for more information.

2.3. Folding and Unfolding the Compact 10 HD

When unfolding the Compact 10 HD, the screen will be set at an ergonomic angle for reading, and the magnifier camera will be activated.

To **open** the Compact 10 HD, follow these steps:

1. Place the Compact 10 HD in front of you so that the screen is facing up, and the Optelec logo is below the screen.
2. Grab and hold the Compact 10 HD in an upright position. Your fingers hold the feet of the stand while your thumbs hold the side of the Compact 10 HD.

3. While supporting the unit by squeezing it with the palms of your hands, curl your fingers underneath the legs and pull them away from the back of the unit. You will feel the legs start to unfold, pull until they are completely straight.

To **close** the Compact 10 HD, follow these steps:

1. Place the Compact 10 HD in front of you so that the screen is facing towards you.
2. Hold the Compact 10 HD at the sides and place your fingers on the supports between the feet and the unit.
3. Gently press against the hinges of the stand until it starts to fold, then continue folding until the feet are flush with the unit. Tilt the screen down until it clicks in position.

2.4. Cleaning the cameras


For an optimal quality image, it is recommended to ensure that the camera lens and object light window are kept clean. A dirty camera window will result in a fuzzy image and may cause the auto-focus camera to focus incorrectly. To clean the cameras, use the provided cleaning cloth or another soft lens cleaning cloth.

Note: Do not use water or cleaning agents to clean the camera window.

2.5. Buttons and Connections

The Compact 10 HD has the following buttons and connections at the left side of the product: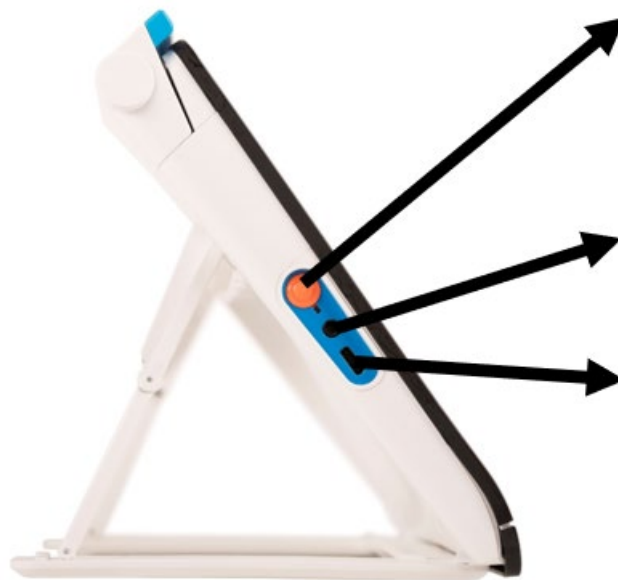

On / Off button

A short press of the orange On / Off button will place the Compact 10 HD into sleep mode or resume the unit. A 6 second press of this button switches the unit off or back on again.

Audio jack

Use the audio jack to connect a headset to the Compact 10 HD.

USB-C connector

Use the USB-C connector to charge the Compact 10 HD. The supplied USB cable connects to this USB port and power supply. Use only the power supply that comes with the Compact 10 HD to power and charge the unit.

3. Cameras

The Compact 10 HD is equipped with three cameras, each serving a specific task or situation. All cameras can be used in Easy mode and Advanced mode. If the image on screen looks out of focus, tap the screen to re-focus the camera and get a sharp image.

3.1. Magnifier camera

The Magnifier camera is used when the Compact 10 HD is in the open position with the stand unfolded. The Magnifier camera is used for reading on a desk or table. The Compact 10 HD is placed on top of the reading material.

3.2. Full Page camera

The Full Page camera is used when the swing-out camera arm is opened. The Full Page camera can be used to view photos and objects more comfortably, or to write your signature, without being restricted in your movements.

Focus lock

The auto focus can be locked for writing or viewing objects at a fixed distance. Use the pen icon at the top right corner of the screen to lock or unlock the focus.

If you own a Compact 10 HD Speech, the full page camera is also used to scan and read full-page documents. Place the document you want to read below the camera, aligning the bottom left corner of the document with the blue line at the right foot of the Compact 10 HD Speech.

3.3. Overview camera

The Overview camera is used when you power on the Compact 10 HD while the Compact 10 HD stand is closed. To switch from Magnifier camera to Overview camera, close the stand and press the On / Off button. The overview camera is optimized to look at objects at a short distance, such as timetables.

4. Using Text-to-Speech & OCR

If you own a Compact 10 HD Speech, you can take a Text-to-Speech snapshot. The Compact 10 HD Speech will extract the text from the image and will read the text aloud.

4.1. Activating the speech

To take a Text-to-Speech snapshot and activate the Speech:

- Position the text or part of the text in the camera view and press the Text-to-Speech snapshot button. When taking a Text-to-Speech snapshot of a full page, use the full-page camera and make sure you align the bottom left corner of the document with the blue indicator at the right side of the stand of the Compact 10 HD Speech. For an optimal result, make sure there is enough light in the environment, prevent shadows and limit the text to only the part of the document you would like to read. If the font you are scanning gets too small, the recognition will lose accuracy and might fail.
- A progress bar will be shown. The Compact 10 HD Speech will start reading after the recognition process has been finished.
- Use the Play/Pause button to start or stop reading.

- Tap and hold (long press) on the text to start reading at the word or section you point at.
- Press the “Mode” button to view the captured text in your selected viewing mode and cycle through the selected high contrast colors of this viewing mode. Please refer to chapter 5.4, View and chapter 5.4 Color for more information.
- If no text is found, a “No text found” message will be shown.

4.2. Adjusting the volume and reading speed

To adjust the volume or reading speed, activate the button bar with a single tap. Tap the Audio button to display the “Volume” and “Speed” sliders. The “Volume” and “Speed” sliders are only present in the snapshot view.

4.3. Adjusting language and voice

To adjust the language or voice in Easy Mode, press and hold the Audio button to go to the Voice menu. To adjust the language or voice in Advanced Mode, press the “Menu” button and then press the “Voice” menu. Refer to Chapter 5.3, Voice for information on navigating the “Voice” menu.

4.4 Table Mode

The regular reading mode reads columns, ideal for newspapers and magazines. But you might occasionally want to read a menu, receipt, table or other text that has a table-like layout. By pressing and holding the Text-to-Speech Snapshot button, the Compact 10 HD Speech will take a snapshot and read all captured text per line from left to right first, and then move to the next line. When you exit the Text-to-Speech snapshot and take another snapshot, you will automatically switch back to the regular reading mode.

5. Easy and Advanced modes


To switch between Easy and Advanced mode, press and hold the Mode button and tap Easy or Advanced.

In Easy mode, you have access to the main features:


- Adjusting the magnification
- Selecting a viewing mode
- Taking a snapshot of an image (Compact 10 HD only)
- Taking a snapshot and use Text-to-Speech (Compact 10 HD Speech only)
- Switching between the Magnifier and Overview camera

In Advanced mode, you have the same functions as in Easy mode, plus some additional features:


- Time and date
- Access to the menu with the options to:
 - Save and open snapshots
 - Changing the language and reading voice
 - Changing color modes
 - Enabling the Reading Line
 - Bluetooth audio and Miracast
 - Other advanced settings
 - Taking a snapshot of an image (Compact 10 HD Speech only)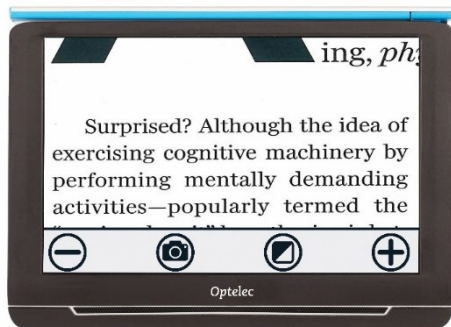


Compact 10 HD Easy Mode

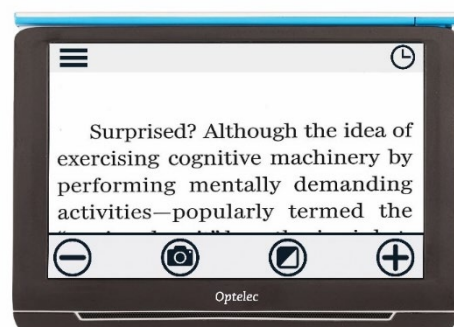Compact 10 HD Advanced Mode

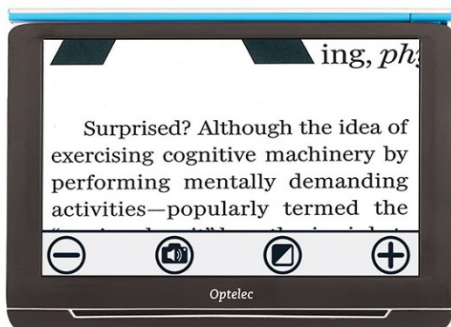Compact 10 HD Speech Easy Mode

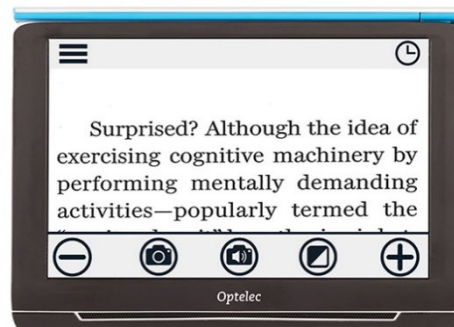Compact 10 HD Speech Advanced Mode

5.1 Features in Easy mode

Adjusting the magnification

To change the magnification, simply tap the screen. The button bar will be shown at the bottom of the screen or at the right side of the screen, depending on the chosen button bar position. The button bar position can be selected in the menu. See chapter 5 for more information on how to change the button bar position. In the button bar tap the + button to increase or the – button to decrease the magnification. Alternatively, use the pinch-to-zoom gesture to change the magnification. Tap the screen to hide the button bar. The button bar will automatically hide when the screen is not touched for 10 seconds.

Selecting a viewing mode

Tapping the Mode button in the middle of the button bar allows you to select one of the following viewing modes:

- **Photo mode:** Viewing full color texts and photos
- **Negative mode:** Reading in high contrast, white on black
- **Positive mode:** Reading in high contrast, black on white
- High contrast **color combination 1**
- High contrast **color combination 2**

Taking a snapshot of an image (Compact 10 HD only)

If you own a Compact 10 HD, press the snapshot button to take a snapshot. You can adjust the magnification and viewing mode of the snapshot. Press the X in the top right corner to close the snapshot and return to live video mode.

Taking a snapshot and use Text-to-Speech (Compact 10 HD Speech)

If you own a Compact 10 HD Speech, press the Text-to-Speech snapshot button to take a snapshot and have it read aloud. Use the Play/Pause button to start or stop reading. You can adjust the magnification and viewing mode, adjust the volume and reading speed and change the voice within the Text-to-Speech snapshot. Press the X in the top right corner to close the Text-to-Speech snapshot and return to live video mode. Refer to Chapter 4 for more information on how to use Text-to-Speech.

Contrast Adjustment

If characters or other on-screen elements are not clearly visible in the high contrast color mode you selected, you can enhance the image and increase the sharpness. To adjust the contrast, press and hold the touch screen with three fingers for approximately three seconds and a slider will appear. Slide to the left or right to adjust the contrast. Press the reset button to go back to the default setting. Tap anywhere on the screen to exit.

5.2. Features in Advanced mode

In Advanced mode, you have the same functions as in Easy mode, plus some additional features.

Time and date

To open the clock, tap the screen and tap on the “Clock” icon at the top right corner of the screen. Use the “Back bar” with the arrow to exit the time and date screen.

To set the time and date, tap the time or date. To change a value, tap on the field you would like to change and tap on the up or down symbol to change the value. Tap on the “Back bar” to exit the set time and date screen.

Taking a snapshot of an image (Compact 10 HD Speech)


If you own a Compact 10 HD Speech and just want to take a snapshot of an image without using Text-to-Speech, press the snapshot button. You can adjust the magnification and viewing mode of the snapshot. Press the X in the top right corner to close the snapshot and return to live video mode.

5.3 Menu

Opening and closing the menu

To open the menu, tap on the screen and tap on the “Menu” icon at the left side of the screen. This will open the main menu. To close the menu, tap on the “Back bar”.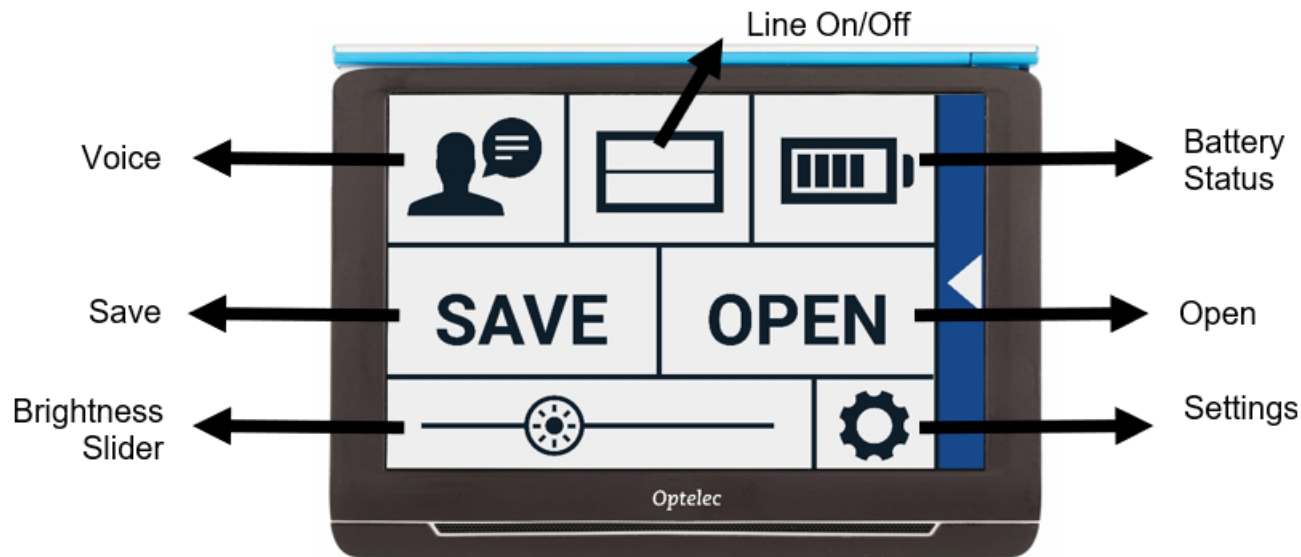

The exact items in the menu will vary, depending on whether you own a Compact 10 HD or 10 HD Speech. To activate a button, tap the button once. Activating a button will open a new menu screen or list. To go back to the main menu, tap on the “Back bar”. To adjust the slider, tap and hold the slider button while moving it to the right or left.

The main menu contains the following options:

- Voice (Compact 10 HD Speech only)
- Line On/Off
- Battery status
- Save
- Open
- Brightness slider
- Settings

Voice (Compact 10 HD Speech only)

To select a voice, language and menu language, or to add or remove a voice or language, open the menu and tap the “Voice” button.

Selecting a voice, language and menu language

This menu item lets you select your reading voice and language. To select another reading voice, tap the voice icon in the top left corner of the main menu to open the list with preselected voices. Each voice represents a language. Tap on a voice to select the voice. If adjusting the reading voice results in a change of language, the menus will automatically change to that language.

Adding or removing a voice or language

To add or remove a voice or language, tap the voice icon in the top left corner of the main menu. In the list with preselected voices, tap on the “Add/Remove” button. To add or remove a reading voice, select the language from the list by tapping on the language. This will display a list with available voices for the selected language. To enable a reading voice, select it from the list by tapping on it. A checkmark in front of the voice name will indicate if it is enabled. When the voice is enabled, it will be available under the “Voice” button in the main menu. By deselecting a voice, the voice will be removed from the available reading voices. The languages of which a reading voice is selected, will appear at the top of the list with languages, and are marked with a checkmark.

Adjusting language and voice in Easy Mode

In Easy mode, you cannot access the Menu. To adjust the language or voice in Easy Mode, press and hold the “Audio” button to go directly to the Voice menu and follow the steps described above.

Line On/Off

The Compact 10 HD can display a reading line to help you position the Compact 10 HD straight on the document. To enable the reading line, tap the “Line” button. Tap this button again to hide the reading line.

Battery status

The battery status is shown in the top right corner of the screen. If you tap the battery icon, a new screen will be opened showing a magnified battery icon and battery status, the “About” button and the “Back arrow” to return to the main menu. The lightning symbol indicates that the battery is charging. Tap on the “About” button in the battery screen to see the application version.

Save

This option is only available in Advanced mode.

To save a snapshot, open the menu and tap the “Save” button.

1. Create a snapshot using the snapshot button.
2. Open the menu.
3. Tap the “Save” button to save the snapshot.

If there is no snapshot to save, the “Save” button is hidden.

Open

To open a snapshot, open the menu and tap on the “Open” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. If there are more snapshots than fit on the screen, swipe up and down to scroll through the list. Tap on the snapshot thumbnail to open it.

Delete

To delete a snapshot, open the menu and tap on the “Open” button. In the open screen, tap on the “Delete” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. Each snapshot has a checkbox at the top right corner. To select a snapshot, tap the checkbox. Tap the “Delete” button to delete the selected snapshots. To delete all snapshots, tap the “Delete” button without selecting a snapshot. You will be prompted to confirm to delete all snapshots. To delete all snapshots, select “Yes”. To cancel deleting all snapshots, select “No”. Both choices will return you to the list of snapshots. Tap the “Back bar” to return to the “Open” screen. Tap the “Back bar” again to return to the main menu.


Adjusting the brightness

Use the slider with the sun icon to increase or decrease the brightness of the screen.

5.4. Settings

Tap on the “Settings” button to enter the Settings menu. The “Settings” button is the button with the gear icon at the bottom right. The Settings menu contains the following buttons:

- Color
- View (Compact 10 HD Speech only)
- Miracast
- Bluetooth (Compact 10 HD Speech only)
- Line
- Buttons
- Theme
- Columns (Compact 10 HD Speech only)
- Lights
- Sound
- Standby
- Language (Compact 10 HD only)
- Restore

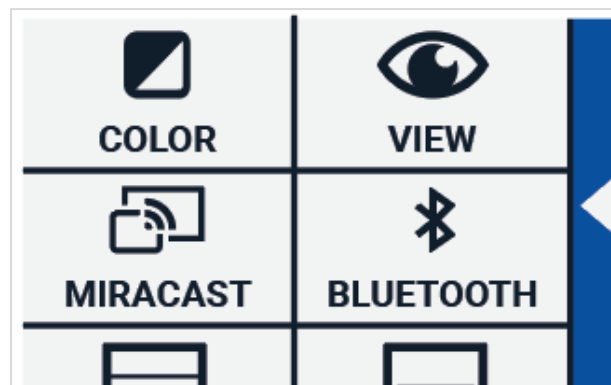Color

In the settings menu, tap the “Color” button to select the preferred high contrast colors. The Compact 10 HD supports four high contrast modes. For each of these four modes, the foreground and background colors can be selected. Three of the high contrast modes can be disabled. The first high contrast mode on the top left is required.

To change the foreground and background colors, tap the up or down arrow next to the mode you want to change. You will cycle through each of the available modes and can stop on the one you prefer. Select the “X” symbol to disable a high contrast mode. Tap on the “Back bar” to return to the settings menu.

View (Compact 10 HD Speech only)

In the settings menu, tap the “View” button to configure how you want to display captured text when using Text-to-Speech.

To configure how all captured text is shown, press the Mode button to cycle through the different viewing modes: Original, Line and Column.

To configure how the word that is read aloud is shown, press the Highlight button to cycle through the different highlight modes: Rectangle and Underline.

When using Text-to-Speech, tap the Mode button in the button bar to view captured text in the viewing mode you just selected and cycle through your configured high contrast colors.

Miracast: connecting wirelessly to a monitor or TV

Miracast is a standard for a wireless connection to a monitor. It can be described as ‘HDMI over Wi-Fi’, replacing the cable from the Compact 10 HD to the monitor or TV. To be able to use the Miracast function of the Compact 10 HD, your monitor or TV needs to support Miracast. Please refer to your monitor or TV manual and documentation to verify if your device supports the Miracast feature.

To connect your Compact 10 HD to a monitor or TV via Miracast, tap the “Miracast” button and switch to Miracast on. A list with available devices will be displayed. Click on a device to connect. It could take up to a minute to establish the connection. Tap on the device again to disconnect. When a Miracast connection is established, both the audio and video will be streamed to the TV or monitor. To guarantee a good connection, the Compact 10 HD needs to be within one meter or three feet of the monitor or TV. Depending on the connection, a short delay of the video and / or audio could be noticed.

Bluetooth (Compact 10 HD Speech only)

To connect a Bluetooth speaker or headphone, tap the “Bluetooth” button and switch Bluetooth on. A list with paired devices is being displayed.

If no speaker or headphone is being paired, tap the “Search” button to search for available devices. Before starting the Bluetooth search, make sure the speaker or headphone is switched on and is set to discoverable. Please refer to the speaker’s or headphone’s manual on how to switch the device to Bluetooth discoverable mode. To pair the speaker or headphone, tap on the device name and if needed enter the pin code. Refer to the speaker’s or headphone’s manual for the pin code of the device.

When the device is paired, to connect the speaker or headphone, tap the device. You will be asked to connect or forget the device. Tap the “Connect” button to make the connection.

To indicate that a device is connected, the connected device will have a checkmark and will be displayed in blue.

To unpair / forget a device, tap the “Forget” button. The device will be removed from the list with paired devices.

Note: Only Bluetooth version 4.0 or higher headsets and speakers are supported.

Line

This menu item enables you to change the reading line position to one of its six positions: horizontal top, horizontal center, horizontal bottom, vertical left, vertical center, vertical right. By default, the reading line is horizontally set to the center of the screen.

Buttons

This menu item enables you to change the button bar position to one of two positions: bottom or side. Depending on your preference, you could change the position of the button bar to the bottom or side of the screen. The default setting is “Bottom”.

Theme

This menu item enables you to select a light or dark theme for the buttons and menus.

Columns (Compact 10 HD Speech only)

This menu item allows you to choose if you want the Compact 10 HD Speech to only read full columns, or incomplete columns as well. By default, **Read full columns only** is selected. If you select to read incomplete columns, not all characters and words are fully captured, which may result in incoherent text.

Lights

This menu item allows you to enable or disable the lights in Magnifier mode and in Overview mode. The Magnifier lights are located to the left and right of the magnifier camera. By default, the reading lights are enabled for an optimal image quality.

The Overview light is located next to the overview camera. By default, the overview light is disabled.

Sound

This menu item allows you to change the volume, mute the audio feedback and enable or disable the vibration feedback. By default, sound is enabled and vibration is disabled.

Standby

This menu item allows you to adjust the standby time. The standby time sets the time after which the Compact 10 HD goes into standby mode to conserve power. Pressing any button will take the unit out of standby mode. By default, the standby time is set to 5 minutes after no button is pressed or no movement of the Compact 10 HD is detected.

Language (Compact 10 HD only)

This menu item allows you to choose the language of the Compact 10 HD. Tap to select your preferred menu language.

Restore

Tap this menu item if you wish to restore the default settings. You will be prompted if you are sure to restore the defaults. To restore the default values, select “Yes”. To cancel restoring the defaults, select “No”.

6. Transferring documents to PC

To transfer snapshots and documents to a PC, you need a Windows 10 PC and the included USB cable and follow the steps below:

1. Switch on the Compact 10 HD and your PC.
2. Connect the Compact 10 HD to your Windows 10 PC using the included USB cable. The Compact 10 HD will briefly show the “Connected to PC” icon.
3. If you connect the Compact 10 HD for the first time, wait until Windows has finished setting up the device.
4. Depending on your Windows settings, Windows Explorer will open the “CP10” device and show the “Internal Shared Storage”.
5. If Windows 10 does not automatically show the Compact 10 HD device, open Windows Explorer and select the CP10 device.
6. Click on the “Internal Shared Storage” and click on the folder “DCIM”. The “DCIM” folder contains folders named with the time and date of the taken pictures. Each folder contains the taken picture and if you own a Compact 10 HD Speech, a .txt document with the text found in the picture. In addition, the folder contains two system files, a JSON and Processed Picture file.
7. Use copy and paste to copy the files to your PC.

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All rights reserved
Optelec Nederland B.V.
Optelec GmbH