


Advanced Mode

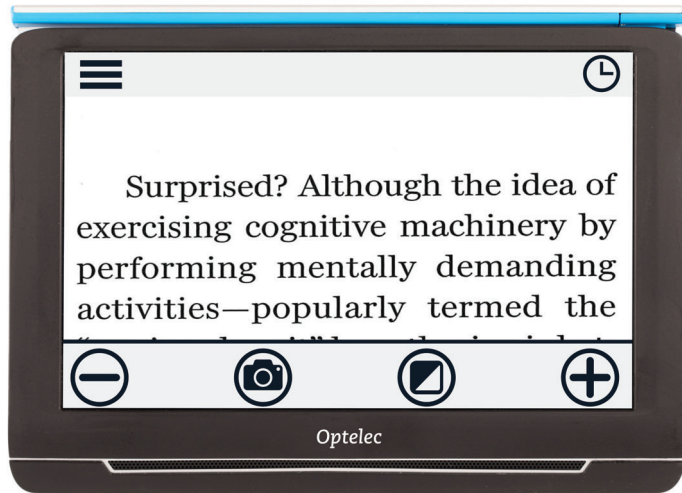


To switch between Easy and Advanced Mode, tap and hold the Mode button and select your choice.

In Advanced Mode, the following buttons are available in addition to the Easy Mode buttons:

 Open menu

 Open clock

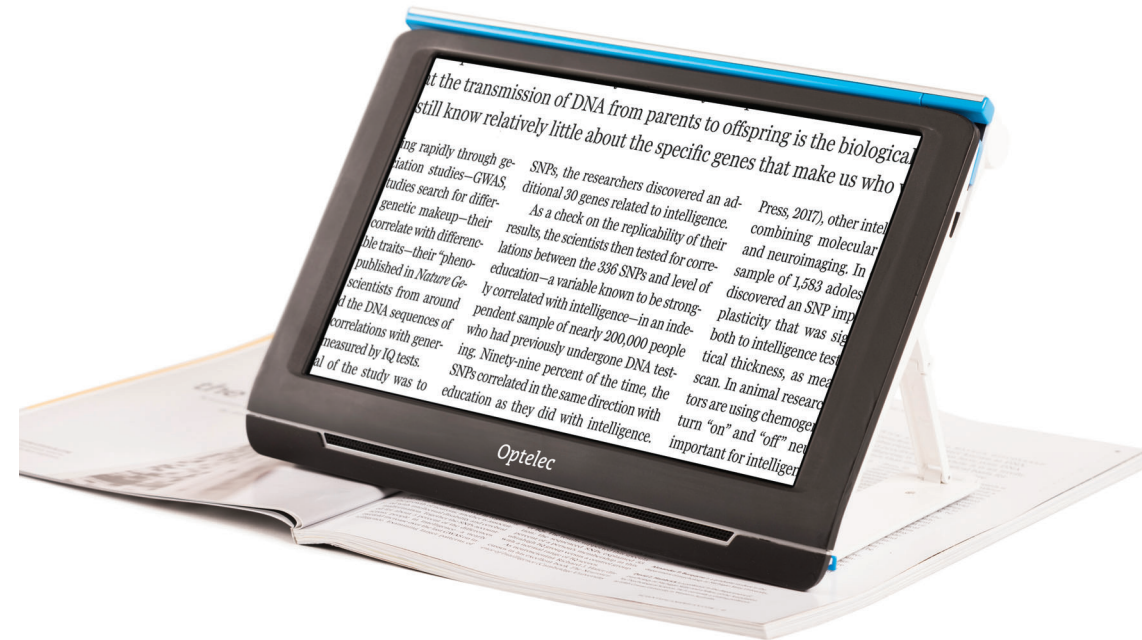

Getting Started



1. Swing-out arm: To view objects and photographs, and to write your signature, open the swing-out camera arm so you will have more space.

2. On / Off button:


Short press: Sleep / Resume.

6 second press: On / Off.

3. Audio jack: Use the audio jack to connect a headset to the Compact 10 HD.

4. USB-C Connector: Charge the Compact 10 HD battery with supplied USB-C cable and power supply. Please charge the battery before initial use.

Easy Mode

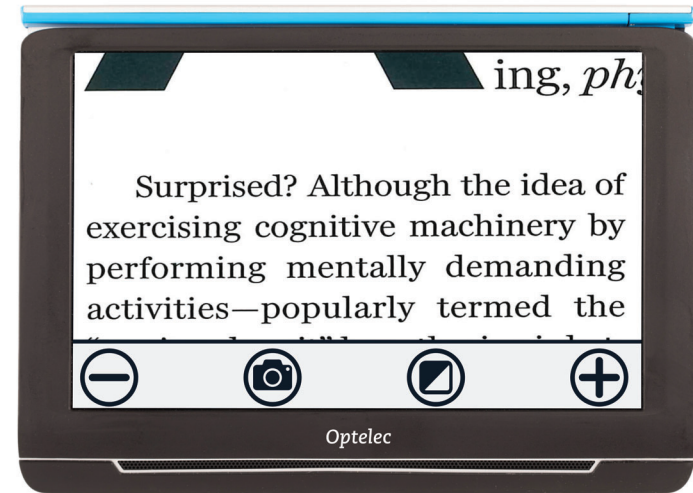

To switch between Easy and Advanced Mode, tap and hold the Mode button and select your choice.

In Easy Mode, the following buttons are available:



Decrease magnification



Take snapshot



Tap: Change viewing mode
Long press: Easy or Advanced mode



Increase magnification