Compact 7 HD

Advanced Features and Functions

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COMPACT 7 HD ADVANCED FEATURES AND FUNCTIONS.....	10
1. INTRODUCTION.....	11
About this manual	12
What's in the box?	12
2. GETTING STARTED	13
Using the Compact 7 HD in the closed or open position.....	14
Cleaning the camera lens and mirror.....	15
Buttons	16
Switching the power on and off.....	19
Adjusting the magnification.....	19
Selecting a viewing mode	20
Using the snapshot button	21
3. CHARGING THE BATTERY	22
Charging the Compact 7 HD.....	22
Standby and Auto Power Off	23
Battery warning.....	23
4. THE COMPACT 7 HD MENU	24
Activating the menu	25
Navigating the menu.....	25
Exiting the menu	25
Menu: Information.....	26
Menu: Images.....	26
Menu: Brightness.....	28

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1. Introduction

Congratulations on choosing the Compact 7 HD!

Optelec's unique auto-focus electronic video magnifier enables you to view and magnify printed text and images in high resolution HD quality in full color or one of the high contrast modes. The Compact 7 HD 7-inch display, combined with its low magnification of 2.5 times, gives you maximum overview while providing up to 30 times magnification for a higher zoom level.

With its multifunctional design, unique lighting concept, autofocus, and integrated reading stand, the Compact 7 HD can be used for reading, viewing images, price tags, displays and much more while at home or on the go.

If you have questions or suggestions concerning the use of this product, please contact your distributor or Optelec's headquarters, using the contact information on the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your Compact 7 HD!

About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and basic operation of the Compact 7 HD. Please read this manual carefully before using your device.

What's in the box?


The Compact 7 HD packaging contains the following:

- The Compact 7 HD video magnifier
- A protective pouch
- A wrist strap
- A power supply⁴ country-specific power plugs (EU, US, UK, and AU)
- A screen cleaning cloth
- *Compact 7 HD Setup and Use*
- *Compact 7 HD Safety and Care*

If any of these items are missing from your package, please contact your distributor.


2. Getting started

The Compact 7 HD is placed correctly in front of you when the screen is facing up, and the large white button and orange on / off switch are at the bottom of the screen.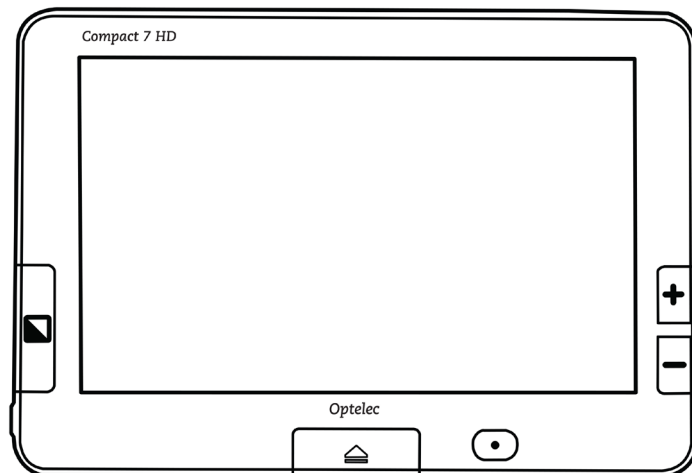


Using the Compact 7 HD in the closed or open position

The Compact 7 HD can be used in two ways: in the closed or open position. In the closed position, hold the unit in your hand and point the camera (located at the back of the unit) to the object or text you wish to view or read. The closed position is ideal for quickly scanning and viewing objects or documents.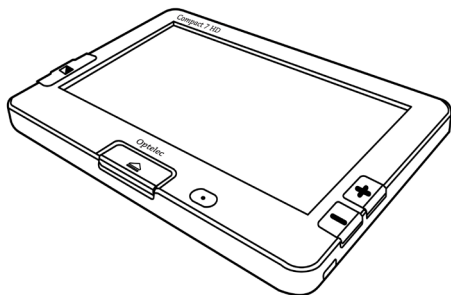

Compact 7 HD closed

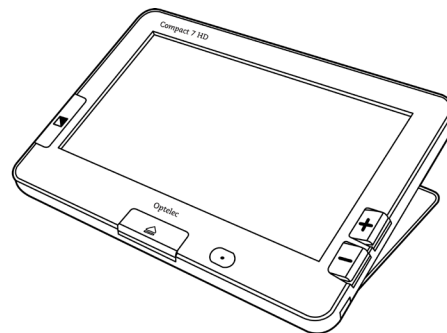
Compact 7 HD open

In the open position, the screen will be set at an ergonomic angle for reading, and the camera will point downward. The open position enables you to read longer texts thanks to the integrated reading stand.

To open the unit, follow these steps:

1. Place the Compact 7 HD in front of you so that the screen is facing up, and the large white button and orange on / off switch are at the bottom of the screen.
2. Press the white-colored button located in the centre at the bottom of the screen. The screen will open, and the Compact 7 HD will switch on.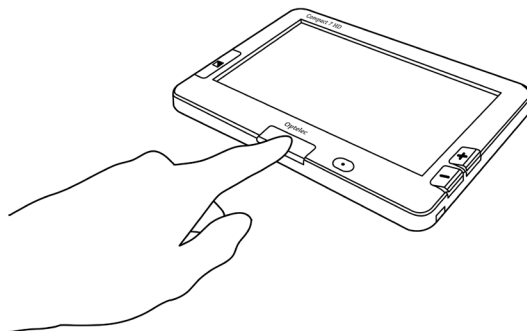


Cleaning the camera lens and mirror

For an optimal quality image, it is recommended to ensure that the camera lens and object light mirror are kept clean. A dirty lens will result in a fuzzy image and may cause the auto-focus camera to incorrectly focus. A clean object light mirror is important for preventing dark or light spots and for ensuring an evenly lit image. To clean the camera lens and mirror, use the provided cleaning cloth or another soft lens cleaning cloth.

Note: Do not use water or cleaning agents to clean the lens and mirror.

Buttons

The Compact 7 HD has the following buttons: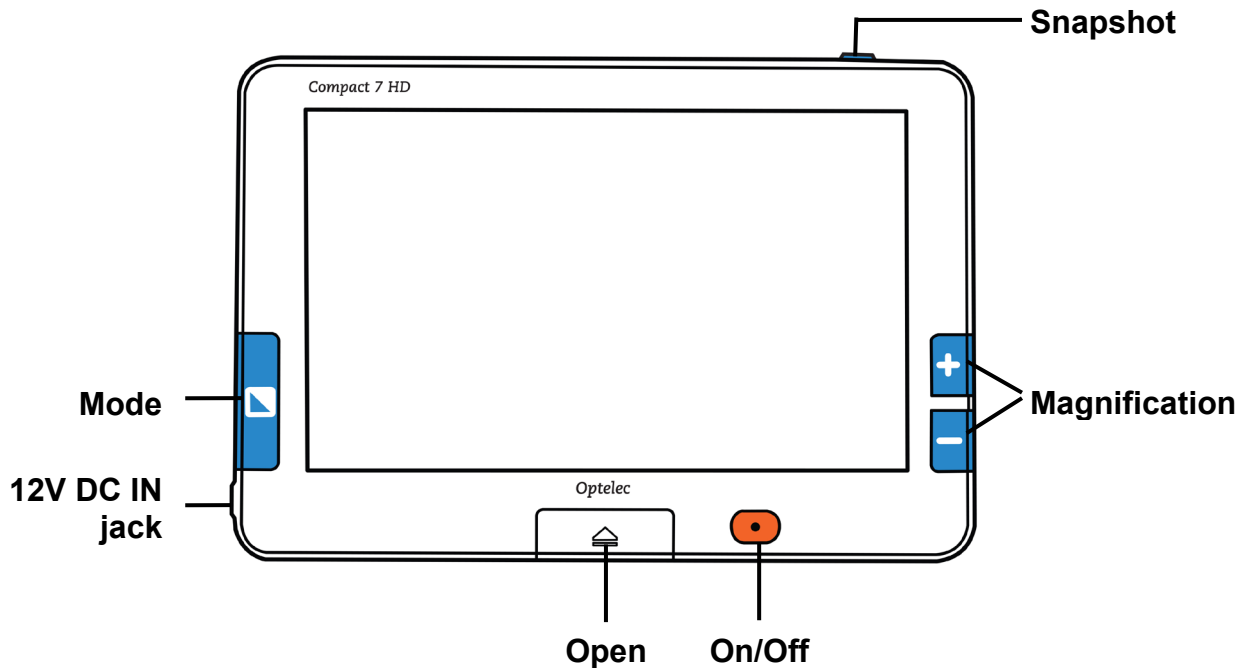

Open

Pressing the white button in the centre at the bottom of the screen will open the Compact 7 HD and switch the unit on.

Mode

Pressing the Mode button to the left of the screen allows you to select one of the viewing modes. Pressing this button for three seconds will turn off the object lighting. Pressing this button for three seconds again, will turn back on the object lighting.

Magnification

Pressing the + or – button to the right of the screen allows you to increase or decrease the magnification. Pressing the + and – buttons simultaneously for one second will allow you to enter or exit the Compact 7 HD menu.

Snapshot button


The snapshot button located on the top right of the unit allows you to take a temporary picture of a distant text or image, enabling you to bring the image closer to your eyes.

On/Off switch

Pressing the oval orange button, located right of the white Open button in the centre at the bottom of the screen, for one second, will switch the unit on and off.

12V DC IN jack

The 12V DC IN jack is located on the left side at the bottom corner of the Compact 7 HD. The power supply cable connects to this jack. Use only the indicated power supply that comes with the Compact 7 HD to power and charge the unit.

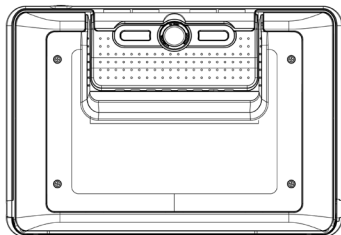The camera of the Compact 7 HD is located at the back of the unit. To the left and right of the camera are two object lights that are used when the unit is in the closed position.

Switching the power on and off

To switch on the Compact 7 HD for the first time, the unit needs to be charged. Once charged, press the orange oval on / off switch located on the right of the white Open button in the centre at the bottom of the screen for one second. The unit will start up in the mode in which it was switched off. To switch off the Compact 7 HD, press the orange oval button again for one second.

When switching on the Compact 7 HD, it will take approximately 3 seconds before an image displays on screen.

The unit can also be switched on by opening the Compact 7 HD as described earlier. This is a default setting. Closing the unit will automatically turn it off. The turn-on-upon-opening and turn-off-upon-closing functions can be enabled or disabled in the menu. Refer to [Chapter 4](#) for more information on how to change these settings.

Adjusting the magnification

To change the magnification settings of the unit, simply press the + or – button located on the right side of the screen. Press the + button to increase or the – button to decrease the magnification. In the open position, when placed on a document, the magnification level starts at 2.5 times and can be increased to 30 times. The Compact 7 HD is equipped with an auto-focus camera. This allows you to use the unit while holding it in your hands in the

closed position. This will provide more overview, but this will decrease the minimum and maximum magnification levels.

Selecting a viewing mode

Pressing the Mode button on the left side of the screen allows you to select one of the following viewing modes:

Photo mode: Viewing full color texts and photos

Negative mode: Reading in high contrast, white on black

Positive mode: Reading in high contrast, black on white

High contrast **color combination 1**

High contrast **color combination 2**

Pressing the Mode button for 3 seconds will toggle the object lighting on or off. Switching modes will automatically turn the object lighting back on.

Using the snapshot button

The snapshot button allows you to take a temporary picture of a text or an image.

- Hold or place the Compact 7 HD over the text or image you wish to capture.
- Press the snapshot button located on the top right side of the unit to take the snapshot.
- Bringing the Compact 7 HD closer to your eyes will enable you to view the object or text closely.
- Press the Mode button to change the viewing mode.
- Use the magnification buttons located to the right side of the screen to change the magnification of the displayed image.
- While viewing the taken image, press the snapshot button for 2 seconds to save the image and return to the live video mode. A short beep and the onscreen indicator will confirm that the image has been saved. A maximum of 75 images can be stored inside the Compact 7 HD memory. A new image will replace the oldest image in memory.

OR

- Press the snapshot button again to and return to the live video mode. It is now possible to take a new snapshot.

Note: If the snapshot button is released before you hear the two beeps, the snapshot will not be saved.

To view or delete snapshots, refer to [chapter 4](#).

3. Charging the battery

Charging the Compact 7 HD

The Compact 7 HD is delivered with a power supply and four country-specific power plugs in EU, US, UK, and AU types. Select the power plug that corresponds to your country and plug it to the power supply.

To charge the Compact 7 HD, connect the power supply to the 12V DC IN jack located on the left side of the unit. Plug the other end of the power supply into a power outlet. The orange LED indicator next to the 12V DC IN jack will glow continuously while the Compact 7 HD is charging. Once the unit is fully charged, the LED indicator will turn green. If the LED indicator flashes, there is a charging problem. In this case, disconnect the power supply and try again.

Charging the Compact 7 HD takes approximately 3.5 hours. A fully charged Compact 7 HD can operate for an average time of 5 hours, depending on the power settings. Please note that, for safety reasons, only the power supply that was delivered with the Compact 7 HD can be used to charge the unit's battery.

Standby and Auto Power Off

The Compact 7 HD will automatically go into standby mode after two minutes if no button is pressed or no image change is detected. Pressing any button will wake up the unit. Four minutes after the Compact 7 HD goes into standby mode, the unit will automatically switch off. Pressing the orange on / off button will switch it back on. The standby time and power-off time can be changed or disabled in the Compact 7 HD menu. Refer to the next chapter on how to change these settings.

Battery warning

When the battery charge reaches below 5% of its total capacity, the unit will issue a battery warning every minute by displaying the battery-empty icon for 2 seconds and by sounding a beep. When this occurs, make sure to charge your unit as soon as possible. When the battery is empty, the unit will automatically switch off.

If a unit does not switch on, please make sure it is charged.

4. The Compact 7 HD Menu

Shows the battery status and firmware version.	
Allows you to view or delete the saved images	
Allows you to adjust the screen brightness.	
Allows you to configure the power settings.	
Allows you to switch the sound on or off.	
Allows you to enable or disable the object lighting settings.	
Allows you to change colors of the 4 high contrast modes.	
Allows you to reset all factory settings to default.	

Activating the menu

To enter the menu, press the + and – buttons simultaneously for two seconds.

Navigating the menu

Use the + and – buttons to navigate between the menu items. The Mode button located to the left of the screen will activate the chosen menu item or confirm the selection. To change a selection, use the + or – button.

Exiting the menu

To exit the menu, press the + and – buttons simultaneously for two seconds.

Menu: Information

This menu item displays the unit's battery status and current firmware version. No changes can be made in this menu.

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- The first option in the menu is Information.
- To exit the menu, press the Mode button or the + and – buttons simultaneously for two seconds.

Menu: Images

This menu contains 3 submenus:

- View: To view saved images
- Delete: To delete saved images
- Delete all: To delete all saved images

Submenu View:

To view the taken images:

- Activate the menu by pressing the + and - buttons for two seconds.
- Select the menu option Images and activate this menu by pressing the Mode button.
- Select the View option. Press the Mode button to confirm your choice.
- Select the image you wish to view by using the + button to move up the list with images, and the - button to move down the list with images.
- Press the Mode button to return to live video mode.

**Submenu Delete**

To delete the taken images:

- Activate the menu by pressing the + and - buttons for two seconds.
- Use the + button to move up and the - button to move down within the menu. Select the menu option Images and activate this menu by pressing the Mode button.
- Select the Delete option. Press the Mode button to confirm your choice.
- Select the image you wish to delete.
- Press the Mode button to delete the image.
- Press the - button to delete the image.
- Press the + button to cancel.
- Press the Mode button to return to live video mode.

Submenu Delete all

To delete all images:

- Activate the menu by pressing the + and - buttons for two seconds.
- Use the + button to move up and the - button to move down within the menu. Select the menu option Images and activate this menu by pressing the Mode button.
- Select the Delete all option. Press the Mode button to confirm your choice.
- Press the - button to delete all images.
- Press the + button to cancel.
- Press the Mode button to return to live video mode.

**Menu: Brightness**

This menu item allows you to adjust the brightness of the display.

- Activate the menu by pressing the + and - buttons simultaneously for two seconds.
- Use the + button to move up or the - button to move down within the menu. Select the menu option Brightness and activate this menu by pressing the Mode button.
- Select the brightness level by using the - button to decrease or the + button to increase the brightness level. There are 5 brightness levels to choose from. The default brightness level is set to 4.
- Press the Mode button to confirm your choice and exit the menu.

Menu: Power


This menu item contains four submenus:

**Standby**


Sets the time after which the Compact 7 HD goes into standby mode to conserve power. Pressing any button will take the unit out of standby mode. By default, the standby time is set to 2 minutes.

**Power off**

Sets the time after which the Compact 7 HD is automatically switched off. By default, the power-off time is set to 4 minutes. These 4 minutes are activated once the default standby time of 2 minutes has expired. By default, the unit is therefore automatically switched off after 6 minutes.

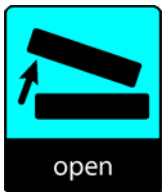**Open**

Enables or disables the automatic power-on function upon opening the Compact 7 HD. By default, the Compact 7 HD will power on upon opening the unit.

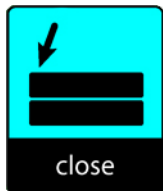**Close**

Enables or disables the automatic power-off function upon closing the Compact 7 HD. By default, the Compact 7 HD will power off upon closing the unit.

Submenu Standby

By default, the standby time is set to 2 minutes. To adjust the standby time of the Compact 7 HD:

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Power and activate this menu option by pressing the Mode button.
- Select the Standby option by using the + or – button. Press the Mode button to confirm your choice.
- Select the time after which the unit will go into standby mode by using the – button to decrease or the + button to increase the time. Selecting DISABLED will disable the automatic standby function.
- Press the Mode button to confirm your choice and return to live video mode.

Submenu Power off

To activate the Compact 7 HD power-off time:

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Power and activate this menu option by pressing the Mode button.
- Select the Power off option by using the + or – button. Press the Mode button to confirm your choice.
- Select the time after which the unit will power off by using the – button to decrease or the + button to increase the time. Selecting DISABLED will disable the automatic power off function.
- Press the Mode button to confirm your choice and return to live video mode.

Submenu Open

To enable or disable the power-on function upon opening the Compact 7 HD:

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Power and activate this menu option by pressing the Mode button.
- Select the Open option by using the + or – button. Press the Mode button to confirm your choice.
- To enable the power-on function upon opening the unit, select the value POWER ON by using the + or – button. Selecting NO ACTION will disable the power-on function upon opening the unit.
- Press the Mode button to confirm your choice and return to live video mode.

Submenu Close

To enable or disable the power-off function upon closing the Compact 7 HD:

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Power and activate this menu option by pressing the Mode button.
- Select the Close option by using the + or – button. Press the Mode button to confirm your choice.
- To enable the power-off function upon closing the unit, select the value POWER OFF by using the + or – button. Selecting NO ACTION will disable the power-off function upon closing the unit.
- Press the Mode button to confirm your choice and return to live video mode.

Menu: Beep

This menu item allows you to enable or disable the feedback tones. By default, the tones are enabled.

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Beep and activate this menu option by pressing the Mode button.
- To enable the sounds, select the value ON by using the + or – button. Selecting OFF will disable the sounds.
- Press the Mode button to confirm your choice and return to live video mode.

Menu: Light

This menu item allows you to enable or disable the object lights - located to the left and right of the camera - for distant viewing. By default, the object lights are enabled.

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Light and activate this menu option by pressing the Mode button.
- To enable the object lights, select the value ON by using the + or – button. Selecting OFF will disable the object lights.
- Press the Mode button to confirm your choice and return to live video mode.

Menu: Color

The Compact 7 HD supports four high contrast modes. For each of these four modes, the foreground and background colors can be selected. Three of the high contrast modes can be disabled if just one high contrast mode is required.

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Color and activate this menu option by pressing the Mode button.
- Select the high contrast mode with the + button to move up or the – button to move down in the submenu Color. Press the Mode button to confirm your choice.

- To select new high contrast colors for the foreground and background, use the + button to move up or the – button to move down within the list of color combinations.
- High contrast modes 2, 3 and 4 can be disabled by selecting the DISABLED value.
- Press the Mode button to confirm your choice and return to live video mode.

Menu: Reset

The Compact 7 HD can be reset to the factory default settings. Any settings that have been adjusted will be restored to factory defaults.

- Activate the menu by pressing the + and – buttons simultaneously for two seconds.
- Use the + button to move up or the – button to move down within the menu. Select the menu option Reset and activate this menu option by pressing the Mode button.
- To restore the factory defaults, select YES by using the + or – button. Choosing NO will leave all settings as they are.
- Press the Mode button to confirm your choice and return to live video mode.

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All rights reserved
U.S.A.
Optelec GmbH