ClearView⁺

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ClearView⁺

Advanced Features and Functions	9
1. Introduction	10
About this manual.....	11
2. Unpacking, Installation and Setup.....	12
3. Operation	14
Operation – Switching ON / OFF	14
Adjusting the magnification	14
Choosing viewing modes	15
Auto-focus lock.....	15
Object lighting ON /OFF	16
Adjusting image control	16
Adjusting the brightness	16
Displaying system information on-screen	16
Locking the reading table	17
Using the footswitch	17
4. Feature Packs	18
4.1 Feature Pack Basic	18
Semi-colors	18
Find function (position locator)	19
Pointer (position locator)	19
4.2 Feature Pack Advanced.....	20
Switching between the ClearView ⁺ and a computer / external source image	20
Activating line markers / window shades.....	21
Sizing the lines markers / window shades	21
Positioning lines markers / window shades	21
Horizontal / vertical button	21
5. Adjusting the Monitor	22
6. Troubleshooting	23
7. Contacting Product Support	24

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ClearView⁺

Advanced Features and Functions

Since 1975, Optelec has been developing and producing innovative products to assist vision-impaired people with their daily activities to maintain and regain independence.

Visit Optelec's website at www.optelec.com for more information on products and services.

Every effort has been made to ensure that the information contained in this user manual is correct at the time of publication. Neither Optelec, nor any of its associates, shall be held responsible for any errors or damages that may result from any incorrect use of the information contained in this manual.

1. Introduction

Congratulations on choosing the ClearView⁺, an auto-focus desktop electronic video magnifier developed by Optelec in the Netherlands. The ClearView⁺ enables you to read, write, view photos, do work and much more.

The ClearView⁺ magnifies and enhances text, objects and images in different colors, brightness and contrast. It is easy to operate and can be used at home, in school or at work.

The ClearView⁺ has a unique ergonomic design that focuses on simplicity. The quality reading table allows users to move text and objects smoothly beneath the camera and it integrates the user-friendly control panel. The auto-focus feature has been incorporated as standard in every ClearView⁺ model. Customized ClearView⁺ monitors can be adjusted to prevent glare and light reflections as well as to ensure a more comfortable reading posture.

This manual helps you become familiar with the ClearView⁺ features and operation. Read this manual thoroughly before setting up and using the ClearView⁺.

Note that the ClearView⁺ is a modular system. Different ClearView⁺ models can be assembled by selecting the various modules. This user manual covers all of the models currently available. A number of features and functions can be added by installing different modules. However it may be possible that your model does not support some of the functions or features described in this manual.

If you have any questions or suggestions concerning the use of this product, contact your Optelec distributor or Optelec's headquarters by consulting the contact information page found at the end of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your ClearView⁺.

About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Download the most up-to-date manual on www.optelec.com at the Support section.

2. Unpacking, Installation and Setup

The ClearView⁺ is a modular system with a standard HDMI output that allows you to connect your ClearView⁺ to any HDMI-compatible monitor or TV.

Your ClearView⁺ consists of the following:

- **A base unit** with a reading table and integrated control panel

- **An electronic box** with the camera module, light sources and input / output connectors. You have the choice between:
 - HD 720P
 - Full HD 1080P (for superior image quality)

- **Customized ClearView⁺ monitor.** The ClearView⁺ offers the following monitor options:
 - 22" high brightness TFT monitor
 - 24" high brightness TFT monitor
 - 24" touch screen Speech monitor

For adjusting the different monitors to your preferred position, refer to [chapter 5](#). These monitors may only be fitted as an integrated part of the unit.

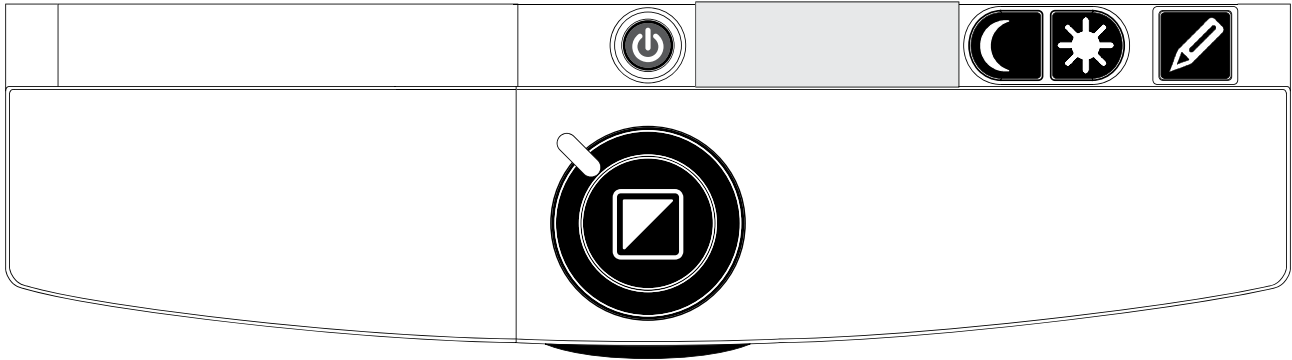
- **Feature Packs** (optional) for additional ClearView⁺ functionality. Optelec offers two feature packs:
 - Feature Pack Basic (with Semi-color, Find and Pointer functions)
 - Feature Pack Advanced (with Semi-color, Find and Pointer functions, Line markers, Window shade and PC-switch functions)

- **Footswitch** (optional) for toggling between the images of the ClearView⁺ and of external sources, such as computers (for Color, HD and PC units).
- **Power supply and cables** for connecting the ClearView⁺ to an electrical wall outlet.
- ***ClearView⁺ Setup and Use***
- ***ClearView⁺ Safety and Care***
- ***ClearView⁺ Installation guide.***

Note: if your ClearView⁺ system was delivered to you in separate modules, unpack each module carefully before proceeding with installation, and consult the accompanying installation guide to install the different modules. Contact your Optelec distributor for further instructions if required.

3. Operation

When using the ClearView⁺, place your hands on the wrist support at the front of the table for easier access to the control buttons, and for steady table movement.

Operation – Switching ON / OFF

To power both the system and monitor, press the ON / OFF button that is located right above the magnification button on the reading table. The ClearView⁺ will start up in the same viewing mode as when it was powered off.

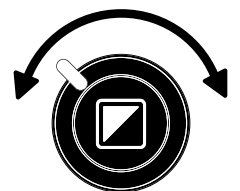
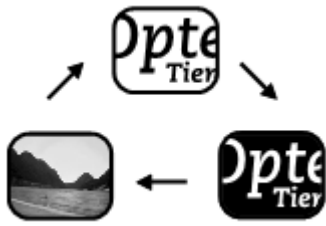
It takes about 10 seconds for the camera unit to initialize before it displays an image. Within about 1 minute, the ClearView⁺ system's light source will achieve full brightness, providing optimal image quality.

Note: a non-ClearView⁺ monitor must be switched on and off separately.

Adjusting the magnification

Locate the zoom button in the center of the table, and turn it clockwise to increase the magnification, and counter-clockwise to decrease it.

Choosing viewing modes

Pressing this button cycles you through 3 different viewing modes:

1. **Photo mode full color** (or gray scale with a monochrome unit)
2. **Reading mode:** This mode enhances the foreground / background contrast. Pictures and text are displayed in high contrast black and white. If an optional Feature Pack was purchased with the ClearView⁺, the background and foreground color combination can be changed.
3. **Reverse reading mode:** This mode reverses the chosen background and foreground color combinations of the reading mode.
4. **Photo mode gray scale** (if enabled)

Note: If the default gray scale setting is enabled on your ClearView⁺, this button also includes a fourth viewing mode, which shows the image in gray scale (true black and white).

Auto-focus lock

The ClearView⁺ features an automatic focus system that produces a sharp on-screen image at any time.

When switching the ClearView⁺ on, the system starts up in the auto-focus mode. Pressing the auto-focus button stops the ClearView⁺ from continuously operating the auto-focus feature and fixes the focus on the object being viewed. This can be useful for tasks, such as writing, painting or crafts.

When the auto-focus feature is in locked mode, a red light on the central control button (mode / zoom) lights up. To turn the auto-focus back on, simply press the auto-focus button again. The red indicator light turns off.

Object lighting ON /OFF

Pressing this button for four seconds turns off the object lighting. You can choose this option to make it easier to read glossy articles such as magazines and photos. Pressing the button once advances the viewing mode by one position and turns the lights back on; or pressing the button for four seconds turns the object lighting back to the same viewing mode.

Adjusting image control

Press and hold the right or left side of this button to adjust the image control. The image control may be adjusted to make the characters or other on-screen elements sharper, if they are not clearly visible.

Note: If the default control setting is changed on your ClearView⁺, pressing this button adjusts the brightness.

Adjusting the brightness

Press the left or right adjustment button while pressing the auto-focus button to adjust the brightness.

Note: If the default control setting is changed on your ClearView⁺, pressing these buttons adjusts the image control.

Displaying system information on-screen

The magnification factor and the system configuration (system modules) can be displayed on-screen. Press

the auto-focus button for four seconds to have this information displayed. Press it again for four seconds to turn this feature off.

Locking the reading table

Lift the red lever located at the front of the table to lock the table in a stable position. This is very useful when writing, painting and doing hobbies and crafts. Unlock the table by simply lifting the lever again.

Note: When the ClearView⁺ power is off, the brake automatically locks into position. When the ClearView⁺ is switched on, the brake unlocks. Make sure to push the reading table backwards to its home position prior to turning off.

Using the footswitch

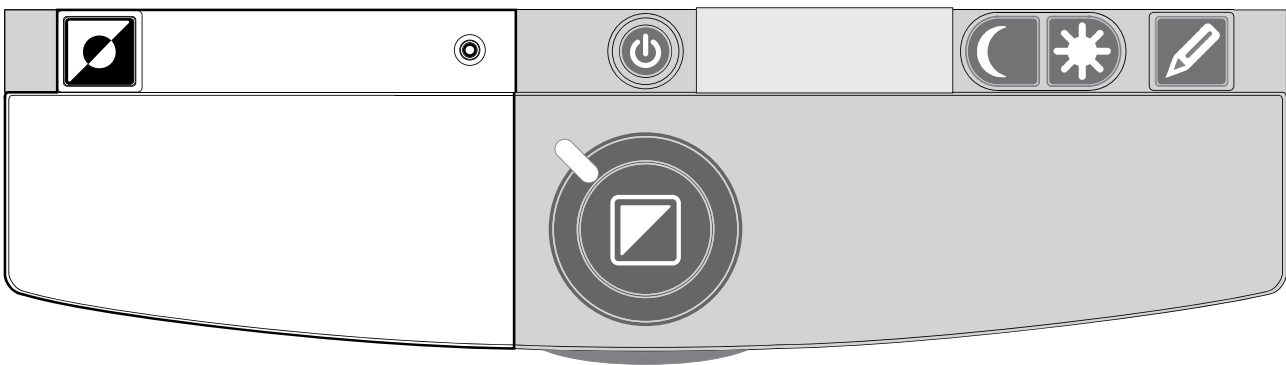
The footswitch allows you to toggle between the ClearView⁺ and of an external source such as a PC (computer).

4. Feature Packs

Feature Packs provide increased functionality to the ClearView⁺. Optelec offers two Feature Packs: a Basic and Advanced Feature Pack.

4.1 Feature Pack Basic

The Feature Pack Basic offers selectable semi-color combinations, grayscale, Full Color+, and the find and position locator.

Semi-colors

Pressing this button in reading mode or reverse reading mode cycles you through 7 different foreground and background color combinations. You may choose to display text in a choice of color combinations when additional contrast is needed, or when a different color combination provides increased visibility. Press and hold the button for two seconds to instantly return to black and white. Press and hold the button again for two seconds to return to your last selected semi-color combination.

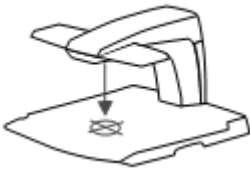
Press this button in photo mode to toggle between the full color, Full Color+ and gray scale mode. In Full Color+ mode, colors are more saturated, which enhances the color image and provides more contrast.

Find function (position locator) ○

The find function provides you with a quick overview of the text, which helps you easily locate the part of a document which is of interest.

Press the position locator button to activate the find function. When the find function is activated, the ClearView⁺ automatically zooms out to the minimum magnification, and displays a crosshair target to indicate the center of the screen. Position the text you wish to read in the crosshair's target by moving the reading table. Once the document has been positioned, press the button again to zoom in on the selected text in your previous magnification setting.


Pointer (position locator) ○

Pressing this button for three seconds activates a red LED light, which shows as a red spotlight on the object in the center of the reading table. It serves to indicate the center of the on-screen image. To assist with writing, place a pen in the red spotlight on the reading table and see the pen displayed in the center of your screen. Press this button again to deactivate the LED light.

4.2 Feature Pack Advanced

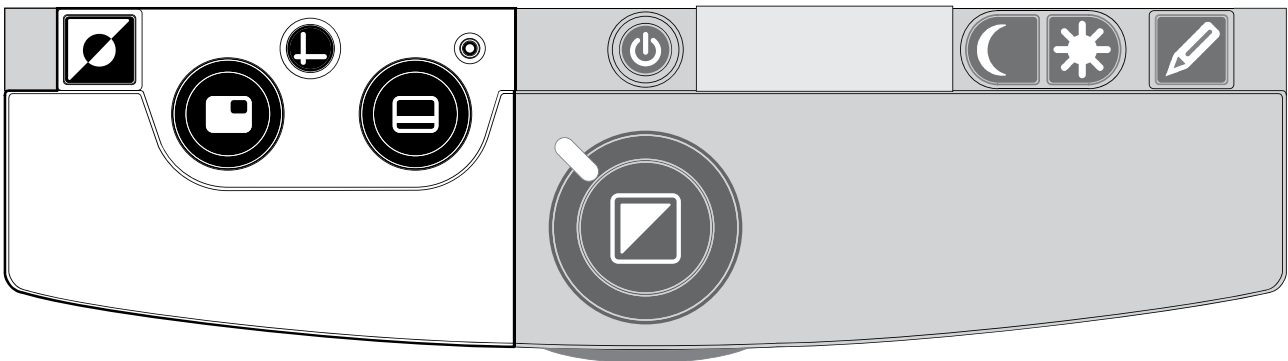The Feature Pack Advanced contains all the features of the Feature Pack Basic and adds the line marker, window shade and PC-switch functions. For a description of the semi-colors and position locator functions, refer to [section 4.1 Feature Pack Basic](#).

Switching between the ClearView⁺ and a computer / external source image

If a computer or another external source is connected to the ClearView⁺, press this button or the footswitch if fitted to toggle between the ClearView⁺ image and the full screen computer image.

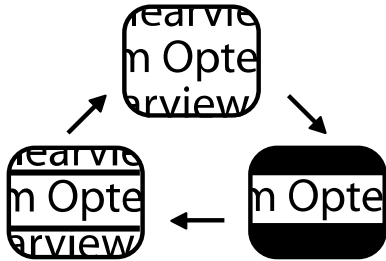
To display the computer image on your ClearView⁺ monitor correctly, make sure that your computer's resolution is set accordingly:

- For the ClearView⁺ 22" TFT monitor, the resolution must be set to 1680 x 1050.
- For the ClearView⁺ 24" TFT monitor or 24" touch screen monitor, set the resolution to 1920 x 1080.

Activating line markers / window shades


Pressing this button cycles you through the line markers, window shades and full screen camera image modes. Line markers are used as guides to make

reading text easier. The window shades function may be used to block a part of an image when screen brightness becomes inconvenient or when needing to concentrate on only one or two lines of text is preferred.

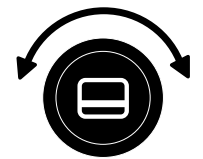Sizing the lines markers / window shades

When line markers or window shades are activated, turning this button narrows or widens the space between the line markers / window edges. This can either be modified horizontally or vertically by using this button .

Positioning lines markers / window shades

When line markers or window shades are activated, turning this button changes their position on screen. Line markers and window shades may be displayed horizontally and vertically depending upon the status of this button described below .

Horizontal / vertical button

Pressing this button allows you to switch the line markers / window shades from the vertical to the horizontal position.

5. Adjusting the Monitor

The ClearView⁺ TFT monitor with standard arm can be adjusted in height and viewing angle for an optimal reading position. Adjusting the monitor assists in preventing glare. Firmly grip the flat panel on both sides to adjust to your preferred position.

The ClearView⁺ monitors equipped with the ultra-flexible TFT arm allows you to adjust the monitor to a preferred position. The monitor can be moved up or down, forward or backward, and tilted horizontally and vertically for optimal ergonomic and viewing comfort. To place the monitor in your preferred position, firmly grip the flat panel on both sides by the surrounding frame, and move or tilt it as required.

Refer to the poster included in the monitor box for more information about adjusting the ultra-flexible TFT arm.

Avoid putting any magnetic or metal objects in front of or near the monitor as they may damage the monitor.

6. Troubleshooting

Manufacturer's default settings

If you experience any difficulty operating the system, reset the factory default settings. Switch the ClearView⁺ off and wait 3 seconds. Then press and hold the On / Off button till the red ring light starts flashing.

No image on the ClearView⁺ monitor after it is switched on

- Check all cables and hardware are connected properly and securely.
- Check that the unit is receiving power by verifying that the green indicator LED light is lit on the power supply.
- Ensure sure the monitor is switched on if a non-ClearView⁺ monitor is connected.

Image is not in focus

- Check the auto-focus lock (the lit red circle around the mode button means that the focus is locked). Press the focus button to return to auto-focus mode.
- Carefully clean the mirror on the camera unit with a non-abrasive cloth.
- Clean the monitor screen with a non-abrasive cloth.

Pictures and text look gray or have low contrast

- Press the mode button to switch to the high contrast reading mode.
- Adjust the image control or brightness level on the control panel.
- If you are using a non-ClearView⁺ monitor, modify the monitor's brightness and contrast settings .

The computer image is not displaying

- Check to see if the Advanced Feature Pack is installed.
- Verify the computer's video (HDMI) connection to the base unit (do not connect the PC directly to the monitor).
- Check the status of the  button.
- When operating with the footswitch, check to see if the Advanced Feature Pack is installed.
- Check the computer for any malfunction by connecting the PC directly into the monitor.
- Make sure the PC resolution is set to 1920x1080.

The reading table does not move or does not move smoothly

- Unlock the table by pulling the lever, located under the front of the table.
- Inspect the tracks for dust or any other obstructions.

7. Contacting Product Support

If you have questions about operating, installing or setting up your ClearView⁺, contact your Optelec distributor or the Optelec office nearest you. See the last pages of this manual for a complete list of Optelec addresses.

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All rights reserved
Optelec GmbH