magnification

MultiView HD

**User manual
Gebruikershandleiding
Manuel d'utilisation
Manual del usuario
Bedienungsanleitung**

Index

1. Introduction.....	7
About this manual	7
2. What's in the Box?	8
3. Getting to Know Your MultiView HD.....	9
Setting up the MultiView HD.....	9
Choosing between left- or right-hand usage.....	10
Positioning the camera head and camera arm	10
Left/Right Rotation	10
Camera positions	11
Close-up viewing.....	12
Self-viewing.....	12
Distance viewing	12
4. Getting Started.....	13
Switching On / Off	13
Setting the level of magnification.....	13
Viewing modes.....	13
Find function	14
Autofocus lock.....	14
Using the Snapshot button.....	14
Activating line markers / window shades	15
Positioning line markers / window shades	15
White Balance.....	15
Brightness adjustment.....	15
Left/Right Rotation	16
Frequency settings.....	16
LED On/Off	16
Appendix A: Safety	17
Appendix B: Technical information.....	18
Appendix C: Warranty and compliances	19
Optelec offices worldwide.....	79

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User Manual

Version 1.1

© 2014 Optelec, the Netherlands.
All rights reserved

Optelec
2993 LT Barendrecht
The Netherlands
Telephone: +31 (0)88 6783 444. Fax +31 (0)88 6783 400
Internet: www.optelec.com

1. Introduction

Congratulations on choosing the MultiView HD, an easily-transportable HD video magnifier from Optelec. Use the MultiView HD for reading, writing, viewing photographs, doing crafts, viewing distant objects, putting on makeup, and so much more!

The MultiView HD enables you to view printed text, images and objects in different colors, and to magnify them up to more than 75 times on a 22" or 24" wide screen display. With its ergonomic control pad, flexible camera arm and optional light-weight X-Y table, the MultiView HD offers superior operating and reading comfort. The MultiView HD can be used at home, at school and in the office.

If you have any questions or suggestions concerning this product, please contact your distributor or the Optelec headquarters by consulting the contact information page found at the end of this user manual. Your feedback is greatly appreciated. We hope you enjoy working with your MultiView HD.

About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This user manual offers a comprehensive guide to the features and operation of the MultiView HD video magnifier. The illustrations in this user manual may be different from the actual situation. Optelec will not assume any responsibility for the misinterpretation of any of the content found in this user manual. Please read this user manual carefully before using your MultiView HD.

2. What's in the Box?

The MultiView HD packaging contains the following items:

- The MultiView HD video magnifier
- A control pad
- A power cable
- A lens cloth
- This user manual

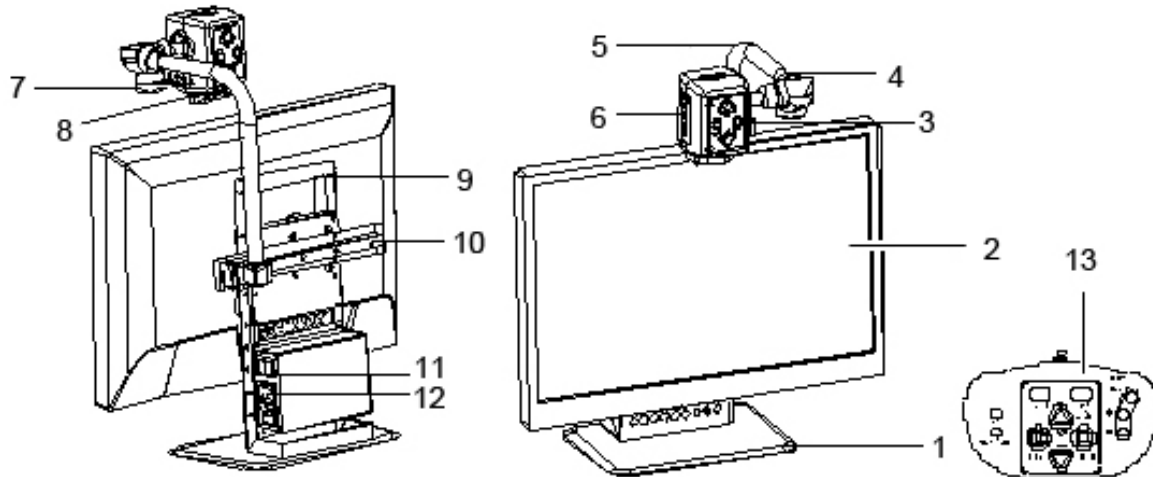
Optional accessories:

- A carrying case for the MultiView 22 inch (not available for the MultiView 24 inch)
- An X-Y reading table

If any items are missing from your package, please contact your local Optelec distributor.

3. Getting to Know Your MultiView HD

The following figure shows the MultiView HD's main components.



- | | |
|----------------------------------|------------------|
| 1. Base stand | 8. Close-up lens |
| 2. 22" / 24" wide screen display | 9. Handle |
| 3. LED On/Off button | 10. Arm Slide |
| 4. Camera arm | 11. Power On/Off |
| 5. Camera Rotation Axes | 12. Power socket |
| 6. Camera head | 13. Control Pad |
| 7. LED light | |

Setting up the MultiView HD

To set up the MultiView HD, please follow the steps described below:

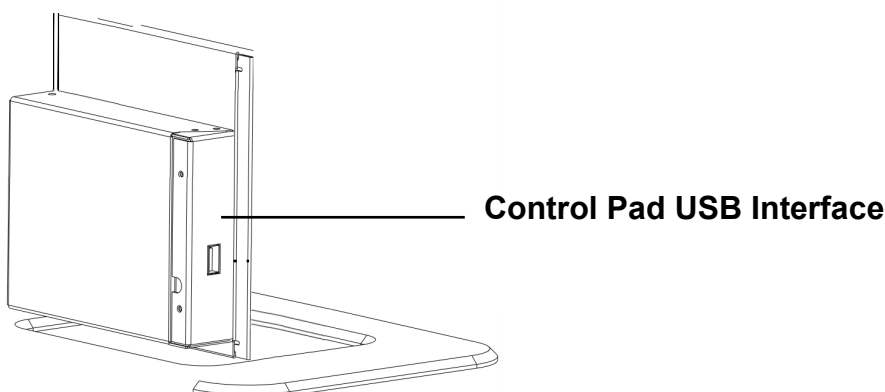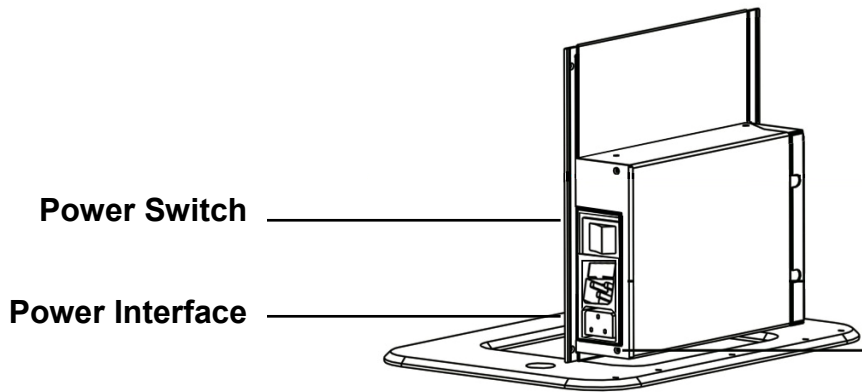
- Connect the control pad in the USB port located on the left side of the base stand.
- Connect the power cable in the power connector located on the right side of the base stand.
- Plug the other end of the power cable into an electrical power outlet.


Choosing between left- or right-hand usage

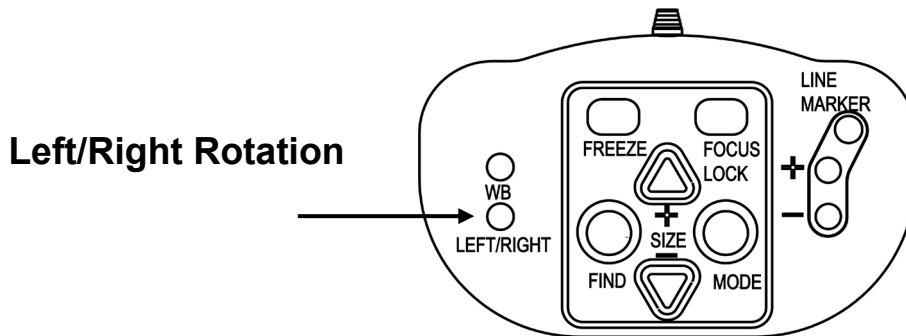The MultiView HD is designed for both left- and right-hand users. To set the unit according to the type of user, position the camera head as described in the next paragraph and rotate the MultiView HD image, if necessary, by pressing the Left/Right Rotation button.


Positioning the camera head and camera arm

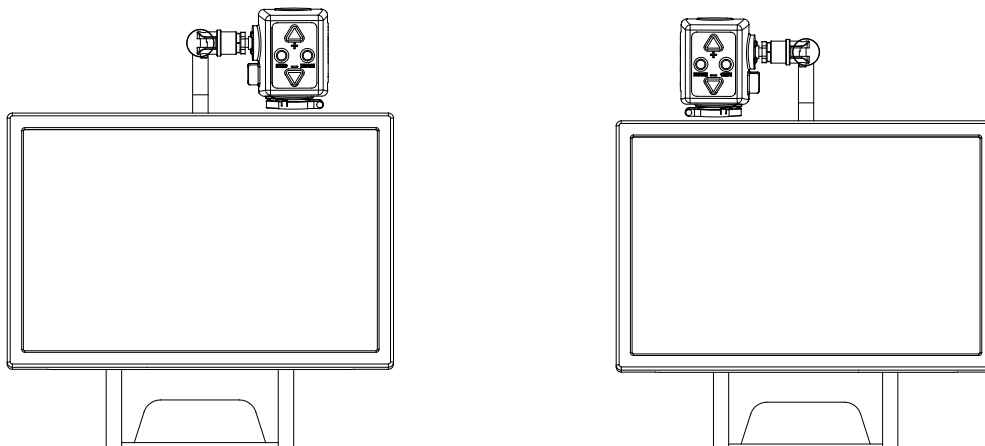The MultiView HD features a flexible camera head and camera arm. The camera head can be rotated 180 degrees at a vertical axis in front of the screen to suit either the left- or right-handed user. The following figure shows the camera head in the position for right- and left-handed users.


Camera position for right-handed user Camera position for left-handed user

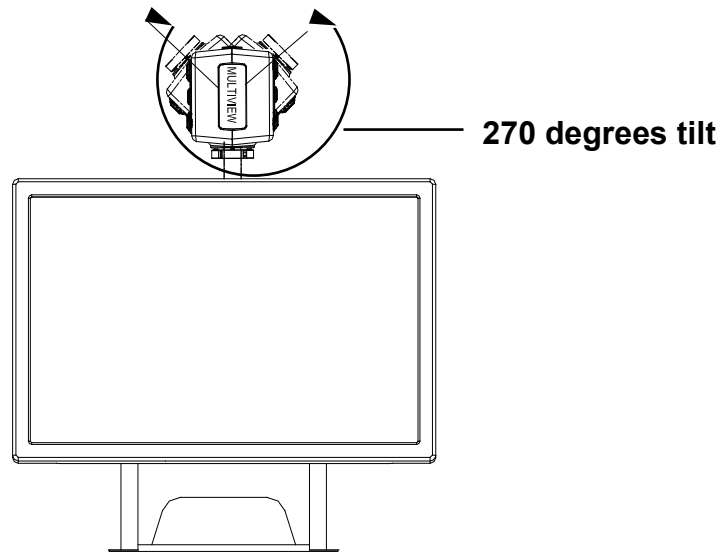At a horizontal axis, the camera head can be tilted 270 degrees. Tilt the camera head and rotate it to view objects further away such as a whiteboard. The following figure shows the different tilt movements.

Note: Do not force the camera head beyond its limit as this could permanently damage the rotation axis feature.

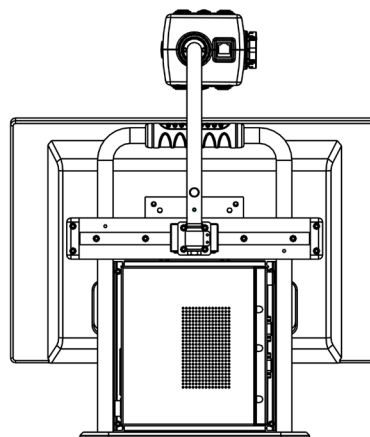The camera arm can be moved from left to right, and from right to left, by using the slider located at the back of the base stand. The following figure shows the slider.


Camera positions

The MultiView HD automatically returns to its original settings when the camera is switched from close-up to distance viewing and vice-versa. For example, when the camera is tilted quickly from the whiteboard you are viewing to the top of your desk, your preferred viewing mode and magnification settings will automatically be applied. Three setting options can be stored: close-up viewing, self-viewing and distance viewing.

Close-up viewing

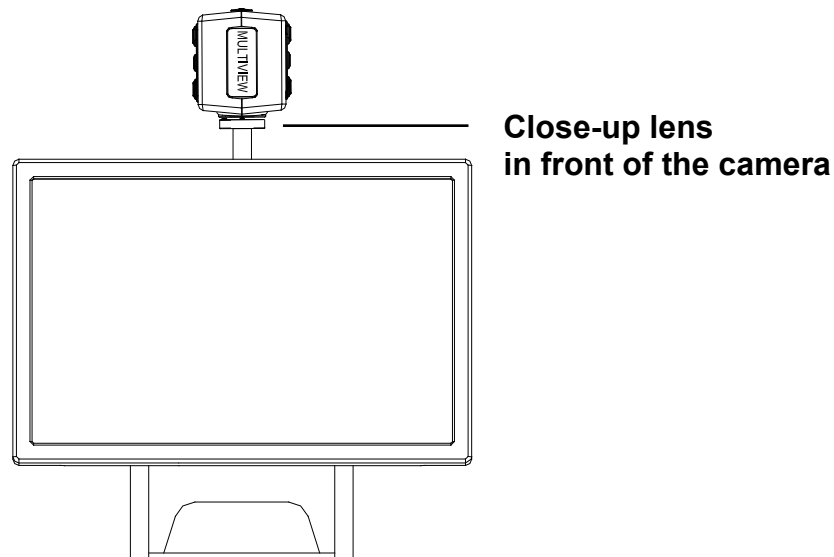To view objects placed on a table, or to write under the camera, point the camera head toward the top of the desk and position the close-up lens in front of the camera. The recommended distance for close-up viewing is between 30 and 40 cm (12 and 16 inches).

Self-viewing

To view yourself and to use the MultiView HD as a mirror, point the camera toward you and position the close-up lens away from the camera. The recommended distance for self-viewing is 40 to 50 cm (16 to 20 inches).

Distance viewing

To view objects further away at a distance between 1 and 5 m (3 and 15 feet), point the camera head toward the object and remove the close-up lens. This viewing mode can be used to read text on a whiteboard.

4. Getting Started

Switching On / Off

Press the On / Off switch located on the right side of the base unit to turn the MultiView HD on or off.

Setting the level of magnification

Press the + and – button located at the centre of the control pad, or on the camera head, to adjust the level of magnification. To increase the magnification, press the + button and to decrease it, press the – button. To adjust the level of magnification in a continuous manner, press and hold down either buttons for a few seconds. The MultiView HD can magnify text and objects from 2 to 81 times their original size (depending on the monitor size) and at a surface distance of 40 cm away from the camera.

Viewing modes

Pressing the Mode button on the control pad or on the camera head will cycle you through 8 different viewing modes:

- **Full Color Photo Mode**
This mode will display text, pictures and objects in full color.
- **Gray Scale Photo Mode**
This mode will display text, pictures and object in shades of gray.
- **Positive Reading Mode (High contrast black on white)**
This mode enhances the foreground / background contrast. Pictures and text will be displayed with a black foreground on a white background.
- **Negative Reading Mode (High contrast white on black)**
This mode reverses the positive image. Pictures and text will be displayed with a white foreground on a black background.
- **High contrast white on blue**
This mode will display white text on a blue background.
- **High contrast yellow on black**
This mode will display yellow text on a black background.
- **High contrast blue on yellow**
This mode will display blue text on a yellow background.
- **High contrast green on black**
This mode will display green text on a black background.

The MultiView HD allows you to customize the 4 high contrast color combinations that you can select in the Mode menu. Pressing the Left/Right Rotation button and the Lines & Windows button simultaneously will display the message “Set” on the top right of the screen. You can now customize the high contrast color combinations.

- Press the Up button to change the foreground color.
- Press the Down button to change the background color.

If you want to restore the default settings, press and hold the Left/Right Rotation button and Mode button on the control pad simultaneously for 3 seconds. Please note that the Full Color Photo Mode, Gray Mode, Negative Reading Mode, and Positive Reading Mode cannot be customized.

Find function

The find function is a useful tool that provides a global view of the document on screen, enabling you to quickly scroll through the text.

Press the Find button on the control pad or on the camera head, and hold it down to activate the find function. When the find function is activated, the MultiView HD will automatically zoom out to its minimum magnification to provide maximum overview. A sign will appear to indicate the centre point of the screen. Move the camera to properly centre your text on screen. Once you have selected your position in the text, release the button. The MultiView HD will automatically zoom in on the portion of text following your selected position, and will revert to your previous magnification setting.

Autofocus lock

The MultiView HD has an autofocus feature that allows you to adjust the clarity of the image on screen. When the MultiView HD is turned on, the system will start up in the autofocus mode.

Press the Autofocus button located on the control pad to lock the focus. This will stop the MultiView HD from continuously readjusting the focus. The message “Focus lock” will appear in the upper right-hand corner of the screen. This is useful for tasks such as writing, painting or doing craftwork. Pressing the Autofocus button again will unlock the focus and the message “Focus lock” disappears.

Using the Snapshot button

The Snapshot button allows you to take a temporary picture of a text or an image. Press the Snapshot button on the control pad to freeze the image. Pressing the Snapshot button again will unfreeze the image.

In Snapshot mode, you have the possibility to pan through images that are zoomed in.

- Hold the WB button and press the + button to move the picture up.
- Hold the WB button and press the – button to move the picture down.
- Hold the WB button and press the Find button to move the picture to the left.
- Hold the WB button and press the Mode button to move the picture to the right.
- Press the Snapshot button again to unfreeze the picture.

Activating line markers / window shades

Pressing the Lines & Windows button on the control pad cycles you through the following options: horizontal line markers, horizontal window shades, vertical line markers, vertical window shades and full screen camera image. Line markers are used as guides to read text more easily. Window shades are used to block parts of the image when the screen is too bright.

Positioning line markers / window shades

When horizontal line markers or window shades are activated, press the Up and Down buttons to change the space size between line markers / window shades.

To widen the space, press the Up button.

To narrow the space, press the Down button.

The MultiView HD allows you to move the line marker and windows up, down, left and right.

- To move the window or lines up or left, press the Line marker and Up button simultaneously.
- To move the window or lines down or right, press the Line Marker and Down button simultaneously.

White Balance

The White Balance is used to adjust the color representation displayed in full color photo mode. Press the WB button on the control pad to cycle through three White Balance settings:

- **Normal color mode**
This color mode displays the more neutral color settings.
- **Warm color mode**
This mode will enhance the warm colors of a picture such as the color red.
- **Cool color mode**
This mode will enhance the cool colors of a picture such as the color blue.

Brightness adjustment

The brightness can be adjusted in High Contrast Mode.

- To increase the brightness, press the WB and Up button simultaneously.
- To decrease the brightness, press the WB and the Down button simultaneously.
- To achieve a more vivid image in a dark environment, please adjust the white balance with the WB button.

Left/Right Rotation

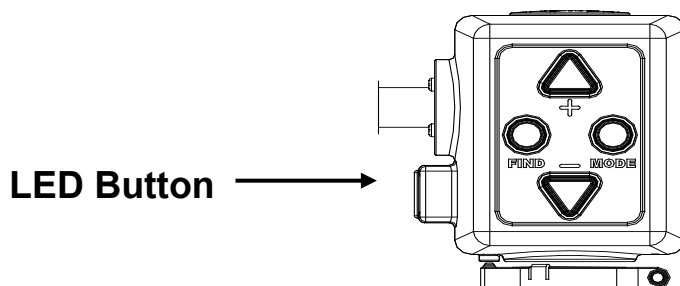
Depending on the camera position, the camera image may be upside down. To rotate the camera image, simply press the Left/Right Rotation button.

Frequency settings

Depending on your country, the preferred frequency is 50Hz or 60Hz. By default, the power frequency is 50Hz in Europe and 60Hz in the US. To switch from 50 Hz to 60Hz or vice versa, press the Mode, +, and – buttons on the camera head simultaneously for 5 seconds.

LED On/Off

By default, the LED light is turned off. Press the red LED button to turn the light on. Press again to turn the light off.

Appendix A: Safety

Safety and Care

- Only use the MultiView HD indoors.
- Do not expose the MultiView HD to extreme or rapid changes in temperature or humidity. Do not place the MultiView HD near a radiator or heater, or where it may be exposed to direct sunlight. These conditions may damage the MultiView HD's electronic or optical components.
- Do not use the MultiView HD near other electrical appliances or inadequately-shielded medical devices as this could cause electromagnetic interferences.
- Do not place the MultiView HD on an unstable surface as it may result in a fall, causing serious damage to the unit or injury to the user.
- Handle the MultiView HD with care. Any rough handling will damage its internal components.
- When moving the MultiView HD to another location, disconnect the power cable along with all other cables from all external connectors. To carry the MultiView HD in a safe and easy manner, use the provided handle located at the back of the unit.
- The reading table and camera arm feature moving mechanical parts. Please ensure your safety when removing or adjusting these mechanical parts (users are not recommended to remove these mechanical parts frequently). Injury could occur if fingers or loose clothing become caught in the operating mechanisms. If you experience any difficulties operating the unit safely, please ask for help by contacting your local Optelec distributor.
- Do not remove any parts from the MultiView HD. Contact your local Optelec distributor to service your MultiView HD, if needed. Do not open any of the components as this will void the warranty. Each component contains parts which should only be serviced by an Optelec repair specialist.
- Always turn the power off when the system is not in use.
- Always unplug the system before cleaning it. Use a soft cloth to clean its external parts. Do not use cleaning agents or abrasive materials as they will damage your unit. Avoid touching the lens of the camera and keep it clean by using proper cleaning means.
- Only use the supplied power cable.
- Do not use the unit in temperatures under +5°C and above +40°C.

Any use of the MultiView HD other than what is described in this user manual will exclude it from warranty conditions.

Appendix B: Technical information

Focus	: Autofocus with focus lock
Monitor	: 22" or 24" LED monitor
Magnification	: 2 x – 75 x (22" monitor) : 2.2 x – 81 x (24" monitor)
Dimensions	: 55 x 44 x 20 cm
Weight	: Unit: approx. 7.5 kg (depending on monitor size)
Viewing modes	: Full color photo mode Gray scale photo mode High contrast black on white High contrast white on black High contrast white on blue High contrast yellow on black High contrast blue on yellow High contrast green on black
Power	: Input 100 - 240 Volt Output 12 Volt / 6 A
Net frequency	: 50 - 60 Hz

Operating conditions

Temperature	: +5°C to +40°C
Humidity	: < 70%, no condensation

Storage & transport conditions

Temperature	: -10°C to +40°C
Humidity	: < 95 %, no condensation

Appendix C: Warranty and compliances

Warranty

Optelec warrants the MultiView HD, effective from the date of delivery, to be free of any defects in material and workmanship.

Warranty is not transferable and does not apply to groups or multi users. The MultiView HD has been designed for the individual purchaser and to be used in a home or office environment. Optelec reserves the right to repair or replace any purchased MultiView HD with a similar or superior product.

In no event shall Optelec or any of its distributors be held liable for any indirect or consequential damages. This warranty is valid only when serviced in the original country of purchase and if seals are intact. For additional warranty claims or for service during or after warranty period, please contact your distributor.

Optelec takes no responsibility for any other use of this device than the use described in this manual.

Compliances

This product complies with the safety and EMC requirements, which fall under the following directives:

Safety

EN 60950-1

EMC

EN 55022

EN 55024

FCC

FCC part 15 Class B

WARNING: The use of accessories, transducers and cables other than those specified and sold by the manufacturer of the MultiView HD as replacement parts for components, may result in increased emissions or decreased immunity of the MultiView HD.

This device bears the CE and FCC approval mark.

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www.optelec.ca**Optelec GmbH**
www.optelec.co.uk**Optelec US Ltd**
U.S.A.