Magnification

Compact 4 HD

User manual
Gebruikershandleiding
Manuel d'utilisation

Manuale utente
Manual del usuario
Bedienungsanleitung

Contents

1. Introduction	12
About this manual	13
What's in the box?.....	13
2. Getting started	14
Using the Compact 4 HD.....	14
Cleaning the camera lens and mirror	16
Buttons.....	17
Switching the power on and off	19
Adjusting the magnification	19
Selecting a viewing mode.....	21
Using the snapshot button.....	21
3. Charging the battery	23
Charging the Compact 4 HD	23
Standby and Auto Power Off.....	24
Battery warning	24
4. The Compact 4 HD Menu	25
Activating the menu.....	26
Navigating the menu	26
Exiting the menu	26
Menu: Information	27
Menu: Brightness	28
Menu: Power	29

Menu: Beeps	31
Menu: Light	32
Menu: Color.....	33
Menu: Reset.....	34
Appendix A: Safety	35
Appendix B: Technical information.....	36
Appendix C: Warranty Conditions	38
Optelec offices worldwide	201

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Internet: www.optelec.com

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 13th February 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserving natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

1. Introduction

Congratulations on choosing the Compact 4 HD!

Optelec's unique auto focus electronic video magnifier enables you to view and magnify printed text and images in high resolution HD quality in full color or one of the high contrast modes. The Compact 4 HD 4.3-inch display, combined with its low magnification of 1.7 times, gives you maximum overview, while providing up to 12 times magnification for those who need a higher zoom level.

With its multifunctional design, unique lighting concept, auto-focus and integrated reading stand, the Compact 4 HD can be used for reading, viewing images, price tags, displays and much more while at home or on the go.

If you have questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information on the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your Compact 4 HD!

About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and basic operation of the Compact 4 HD. Please read this manual carefully before using your device.

What's in the box?


The Compact 4 HD packaging contains the following:

- The Compact 4 HD video magnifier viewer
- The Compact 4 HD stand
- A protective pouch for the Compact 4 HD Viewer
- A wrist strap
- A power supply
- 4 country specific power plugs (EU, US, UK and AU),
- A screen cleaning cloth
- A user manual

If any items are missing from your package, please contact your distributor.

2. Getting started

The Compact 4 HD is placed correctly in front of you when the screen is facing up and the Optelec logo is at the bottom of the screen.

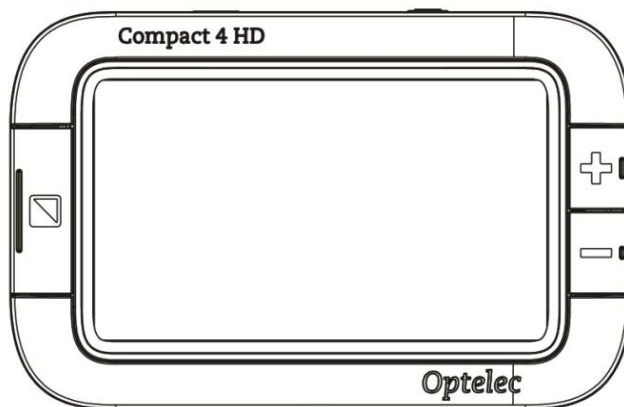Using the Compact 4 HD

The Compact 4 HD can be used in two ways: Use the Compact 4 HD viewer with or without the stand.

Compact 4 HD viewer (without the stand)

Compact 4 HD viewer with the stand

To use the Compact 4 HD viewer without the stand, hold the viewer in your hand and point the camera (located at the back of the unit) to the object or text you wish to view or read. The distance between the camera and object or text needs to be between 5 cm (2 inch) and 30 cm (12 inch) for an optimal image quality.


The Compact 4 HD viewer is ideal to quickly scan and view objects or documents.

When the Compact 4 HD viewer is placed on the stand, the screen will be set in an ergonomic angle for reading with the camera pointing downwards. The stand makes it

extremely easy to hold the Compact 4 HD in your hands and move the unit over the text, enabling you to read longer texts.

To place the Compact 4 HD viewer on the stand, follow these steps:

1. Place the stand in front of you
2. Place the Compact 4 HD viewer on top of the stand so that the screen is facing up and the Optelec logo is at the bottom of the screen. The magnets in the stand will automatically position the Viewer correctly onto the stand.

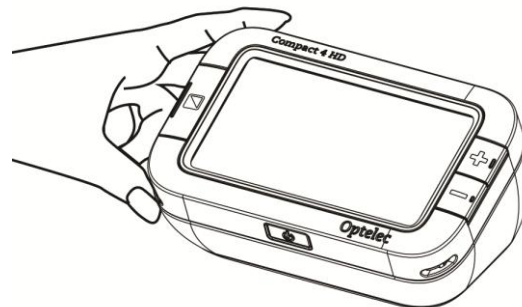Cleaning the camera lens and mirror

For an optimal quality image, it is recommended to ensure that the camera lens is kept clean. A dirty lens will result in a fuzzy image and may cause the auto focus camera to incorrectly focus. To clean the camera lens, use the included cleaning cloth or another soft lens cleaning cloth.

Note: Do not use water or a cleaning agent to clean the lens.

Buttons

The Compact 4 HD has the following buttons:

Mode

Pressing the Mode button to the left of the screen allows you to select one of the viewing modes. Pressing this button for three seconds will turn off the object lighting.

Magnification

Pressing the + or - button to the right of the screen for one second allows you to increase or decrease the magnification. Pressing the + and - buttons simultaneously for one second will allow you to enter or exit the Compact 4 HD menu.

Snapshot button

The snapshot button located on the top right of the unit allows you to take a temporary picture of a distant text or image, enabling you to bring the image closer to your eyes.


On/Off Switch

Pressing the grey button with orange icon, located at the centre of the front side of the Compact 4 HD, for one second, will switch the unit on and off.

5V DC IN jack

The 5V DC IN jack is located on the top side, left of the snapshot button. The power supply cable connects to this jack. Use only the power supply that comes with the Compact 4 HD to power and charge the unit.

The camera of the Compact 4 HD is located at the back of the unit.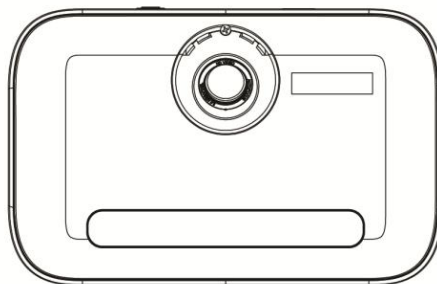

Switching the power on and off


To switch the Compact 4 HD on for the first time, the unit needs to be charged. Once the unit is charged, press the grey on / off button with the orange icon, located at the centre of the front side of the Compact 4 HD for one second. The unit will start up in the mode in which it was switched off. To switch off the Compact 4 HD, press the orange button again for one second.

When switching on the Compact 4 HD, it will take approximately 3 seconds before an image is displayed on the screen.

Adjusting the magnification

To change the magnification, simply press the + or – buttons located on the right side of the screen. Press the + button to increase or the – button to decrease the magnification. When placed on the stand, the magnification range starts at 1.7 times and can be increased to 12 times. The Compact 4 HD is equipped with an auto-focus camera. This allows you to use the Compact 4 HD Viewer without the stand while holding the Viewer in your hands. This will provide more overview, but will decrease the minimum and maximum magnification, depending on the distance of the viewer to the object or text.

Note: in the smallest magnification of 1.7 times, text lines may be displayed with a slight curve. This will be most visible when reading small texts. This “fish eye” effect is caused by the ultra wide angle of the lens.

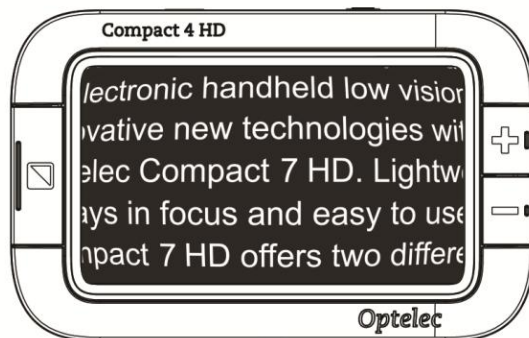Fish-eye effect in the smallest magnification

Selecting a viewing mode

Pressing the Mode button to the left side of the screen, allows you to select one of the following viewing modes:

Photo mode: view full color texts and photos

Negative mode: reading in high contrast white on black

Positive mode: reading in high contrast black on white

High contrast **color combination 1**

High contrast **color combination 2**

Pressing the Mode button for 3 seconds will toggle the object lighting on or off. Switching modes will automatically turn the object lighting back on.

Using the snapshot button

The snapshot button allows you to take a temporary picture of a text or an image.

- Hold or place the Compact 4 HD over the text or image you wish to take a snapshot of.
- Press and hold the snapshot button located on the top right side of the unit.
 - o You will hear a beep when pressing the snapshot button.
 - o Keep the snapshot button pressed.

- The Compact 4 HD will display a camera icon and take a snapshot and sound two beeps.
- Release the snapshot button.
- The Compact 4 HD will display the picture taken.
- The image will be temporarily stored inside the Compact 4 HD memory. Bringing the Compact 4 HD closer to your eyes will enable you to view the object or text closely.
- Use the magnification buttons located to the right side of the screen to change the magnification of the displayed image.
- Press the Mode button to change the viewing mode.
- Press the snapshot button again to unfreeze the image. It is now possible to take a new snapshot.

Note: if the snapshot button is released prior to sounding the two beeps, no snapshot will be taken and the Compact 4 HD will revert back to its original magnification level and viewing mode (those that were set before the snapshot button was pressed).

3. Charging the battery

Charging the Compact 4 HD

The Compact 4 HD is delivered with a power supply and four country-specific power plugs. The power plugs are available in EU, US, UK and AU versions. Select the power plug appropriate to your country and mount this onto the power supply.

To charge the Compact 4 HD, connect the power supply to the 5V DC IN jack, located at the left side of the unit. Plug the other side of the power supply into a power outlet. The orange LED indicator next to the 5V DC IN jack will glow continuously while the Compact 4 HD is charging. When the unit is fully charged, the LED indicator will turn green. If the Orange LED indicator flashes, there is a charging problem. In this case, disconnect the power supply and try again. In a warm environment, or when the Compact 4 HD is switched on while charging, charging may temporarily be paused. *If the charging is being paused, the LED indicator will flash green. The Compact 4 HD will automatically resume charging when the charging protection mechanism detects a temperature drop. Switching the Compact 4 HD off will help to lower the temperature to resume charging the battery.*

Charging the Compact 4 HD takes approximately 3.5 hours. A fully charged Compact 4 HD can operate for an average time of 3 hours, depending on the power settings. Please note that, for safety reasons, only the power supply that was delivered with the Compact 4 HD can be used to charge the unit's battery.

Standby and Auto Power Off

The Compact 4 HD will automatically go into standby mode after two minutes if no button is pressed or no image change is detected. Pressing any button will wake the unit up again. Four minutes after the Compact 4 HD goes into standby mode, the unit will automatically switch off. Pressing the orange on / off button will switch it back on again. The standby time and power off time can be changed or disabled in the Compact 4 HD menu. Please refer to the next chapter on how to change these settings.

Battery warning

When the battery charge is below 5% of its total capacity, a battery warning will be issued every minute by displaying the battery empty icon for 2 seconds and sounding a beep. When this occurs, be sure to charge your unit as soon as possible. When the battery is empty, the unit will automatically switch off.

If a unit does not switch on, please make sure it is charged.

4. The Compact 4 HD Menu

Shows the battery status and firmware version.

Allows you to adjust the screen brightness.

Allows you to configure the power settings.

Allows you to switch the sound on or off.

Allows you to enable or disable the object lighting settings.

Allows you to change the colors of the 4 high contrast

Allows you to reset all factory settings to default.

Activating the menu

To enter the menu, press the + and – buttons simultaneously for one second.

Navigating the menu

Use the + or – buttons to navigate between the menu items. The Mode button located to the left of the screen will activate the chosen menu item or confirm the

selection. To change a selection, use the + or – buttons.

Exiting the menu

To exit the menu, press the + and – buttons simultaneously for one second.

Menu: Information

This menu item shows the battery status and the current firmware version of the unit. In this menu, it is not possible to make any changes.

- Activate the menu by pressing the **+** and **-** buttons simultaneously for one second.
- The first option in the menu is Information.
- To exit the menu, press the Mode button or the **+** and **-** buttons simultaneously for one second.

Menu: Brightness

This menu item allows you to adjust the brightness of the display.

- Activate the menu by pressing the **+** and **-** buttons simultaneously for one second.
- Use the **+** button to move up and the **-** button to move down within the menu. Select the menu option Brightness and activate this menu by pressing the Mode button.
- Select the brightness level by using the **-** button to decrease and the **+** button to increase the brightness level. There are 5 brightness levels to choose from. The default brightness level is 5.
- Press the Mode button to confirm your choice and exit the menu.

Menu: Power

This menu item contains two submenus:

**Standby**

Sets the time before the Compact 4 HD goes into standby mode to conserve power. Pressing any button will take the unit out of standby mode. By default, the standby time is set to 2 minutes.

**Power off**

Sets the time before the Compact 4 HD is automatically switched off. By default, the power off time is set to 4 minutes. These 4 minutes are activated after the default standby time of 2 minutes. By default, the unit is therefore automatically switched off after 6 minutes.

Submenu Standby

By default, the standby time is set to 2 minutes. To adjust the standby time of the Compact 4 HD:


- Activate the menu by pressing the + and – buttons simultaneously for one second.

- Use the **+** button to move up and the **-** button to move down within the menu. Select the menu option Power and activate this menu by pressing the Mode button.
- Select the Standby option by using the **+** and **-** buttons to scroll through the options. Press the Mode button to confirm your choice.
- Select the time before the unit will go into standby by using the **-** button to decrease and the **+** button to increase the time. Press the Mode button to confirm your choice. Selecting DISABLED will disable the automatic standby function.
- Press the Mode button to confirm your choice and exit the menu.

Submenu Power off

To activate the Compact 4 HD power off time:

- Activate the menu by pressing the **+** and **-** buttons simultaneously for one second.
- Use the **+** button to move up and the **-** button to move down within the menu. Select the menu option Power and activate this menu by pressing the Mode button.
- Select the Power off option by using the **+** and **-** buttons to scroll through the options. Press the Mode button to confirm your choice.
- Select the time before the unit will power off by using the **-** button to decrease and the **+** button to increase the time. Press the Mode button to confirm your choice. Selecting DISABLED will disable the automatic power off function.
- Press the Mode button to confirm your choice and exit the menu.

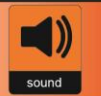Menu: Beeps

This menu item allows you to enable or disable the feedback tones. By default, the tones are enabled.

- Activate the menu by pressing the + and – buttons simultaneously for one second.
- Use the + button to move up and the – button to move down within the menu. Select the menu option Sound and activate this menu by pressing the Mode button.
- To enable the beeps, select the value ON by using the + and – buttons. Press the Mode button to confirm your choice. Selecting OFF will disable the beeps.
- Press the Mode button to confirm your choice and exit the menu.

Menu: Light

This menu item allows you to enable or disable the object lighting of the viewer when used without the stand. By default, the object lighting is enabled.

- Activate the menu by pressing the + and – buttons simultaneously for one second.
- Use the + button to move up and the – button to move down within the menu. Select the menu option Light and activate this menu by pressing the Mode button.
- To enable the object lighting, select the value ON by using the + and – buttons. Press the Mode button to confirm your choice. Selecting OFF will disable the object lighting.
- Press the Mode button to confirm your choice and exit the menu.

Menu: Color

The Compact 4 HD supports four high contrast modes. For each of these four modes, the foreground and background colors can be selected. Three of the high contrast modes can be disabled if just one high contrast mode is required.

- Activate the menu by pressing the + and – buttons simultaneously.
- Use the + button to move up and the – button to move down within the menu. Select the menu option Color and activate this menu by pressing the Mode button.
- Select the high contrast mode with the + button to move up and the – button to move down in the submenu Color. Press the Mode button to confirm your choice.
- To select a new high contrast color for the foreground and background, use the + button to move up and the – button to move down in the list of color combinations. Select a color combination by pressing the Mode button.
- High contrast modes 2, 3 and 4 can be disabled by selecting the DISABLED value.
- Press the Mode button to confirm your choice and exit the menu.

Menu: Reset

The Compact 4 HD can be reset to the factory default settings. Any settings that have been adjusted, will be restored to the factory defaults.

- Activate the menu by pressing the + and – buttons simultaneously for one second.
- Use the + button to move up and the – button to move down within the menu. Select the menu option Reset and activate this menu by pressing the Mode button.
- To restore the factory defaults, select YES by using the + and – buttons. Choosing NO will leave all settings as they were.
- Press the Mode button to confirm your choice and exit the menu.

Appendix A: Safety

- Do not expose the Compact 4 HD to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Compact 4 HD. Contact an Optelec authorized service organisation in your area.
- To avoid risk of electrical damage, keep your Compact 4 HD away from fluids and chemicals.
- Handle the Compact 4 HD with care. Rough handling will damage the internal components.
- Do not use the Compact 4 HD near inadequately shielded medical devices.
- Do not try to open the battery compartment or try to remove the battery.
- Contact your dealer to service this equipment if necessary. **Do not disassemble your unit** as it will void the warranty.
- Always unplug the system before cleaning. Use a soft dry cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.

Use of the Compact 4 HD other than described in this manual will exclude it from warranty conditions.

Appendix B: Technical information

Magnification	1.7 – 12 times
Viewing modes	Full color photo mode High contrast white on black High contrast black on white High contrast color combination 1 High contrast color combination 2
Focus	Auto-focus
Screen	4.3" TFT, 480 x 270
High contrast ratio	800:1
High brightness screen	500 cd/m ²
Dimensions without Stand	130 x 82.5 x 27.5 mm / 5.1 x 3.2 x 1.1 inches
Dimensions with Stand	130 x 94 x 73 mm / 5.1 x 3.7 x 2.9 inches
Battery	average 3 hours continuous use 3.5 hours charging time
Li-ion battery	rechargeable, 3.7V, 2800 mAh or 3240mAh
Weight Viewer	248 grams / 8.7 ounces
Weight Sand	140 grams / 4.9 ounces
Power supply input	100-240V, 50-60Hz, 0.3A
Power supply output	DC5V-2A

Operating conditions

Temperature	+10°C to 35°C / 50°F to 95°F
Humidity	< 70%, no condensation
Altitudes	up to 3000 m (9842 feet)
Pressure	700 – 1060 hPa

Storage & transportation conditions

Temperature	+10°C to 40°C / 50°F to 104°F
Humidity	< 95%, no condensation
Altitudes	up to 12192 m (40000 feet)
Pressure	186 – 1060 hPa

Appendix C: Warranty Conditions

Optelec warrants the Compact 4 HD, effective from the date of delivery, to be free of any defects in material and workmanship.

The warranty is not transferable and does not apply to groups, multi-users and agencies. The Compact 4 HD was designed for the individual purchaser to be used at home and on-the-go. Optelec reserves the right to repair or replace any purchased Compact 4 HD with a similar or improved product.

In no event shall Optelec or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of the Compact 4 HD. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after warranty period, please contact your Optelec distributor.

Optelec is not responsible for any uses of this device other than those described in this manual. Use of the Compact 4 HD other than described in this manual will exclude it from warranty conditions.

RoHS Compliance

This product is in compliance with Directive 2002/95/EC of the European Parliament and of the Council of 27 January 2003, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

European Union Notice

Products with CE marking comply with the EMC Directive (2004/108/EC), and the Low Voltage Directive (2006/95/EC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following international standards:

EMC

EN 55022

EN 55024

FCC

FCC part 15 Class B

Safety

UL 60950-1

WARNING: The use of accessories, transducers and cables, with the exception of transducers and cables sold by the manufacturer of the Compact 4 HD as replacement parts for components, may result in increased emissions or decreased immunity of the Compact 4 HD.

This device bears the CE and FCC approval mark

Federal Communications Commission (FCC) Statement:

This Equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio/TV technician for help.

WARNING: You are cautioned that changes or modifications not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received; including interference that may cause undesired operation.

WARNING: Risk of explosion if the battery is replaced by an incorrect type. Please dispose of used batteries according to the disposal and recycling regulations that are applicable in your country or region!

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All rights reserved
Optelec Limited
Optelec GmbH