Magnification

Compact 6 HD

Compact 6 HD Speech

User manual
Gebruikershandleiding
Manuel d'utilisation

Manuale utente
Manual del usuario
Bedienungsanleitung

Optelec

Contents

1. INTRODUCTION	17
ABOUT THIS MANUAL	17
WHAT’S IN THE BOX?	18
2. GETTING STARTED	19
USING THE COMPACT 6 HD WITH THE READING STAND	20
CLEANING THE CAMERA WINDOW	23
BUTTONS.....	24
POWER ON AND OFF	25
SLEEP AND RESUME	25
THE MAGNIFIER AND OVERVIEW CAMERAS	26
FOCUS LOCK.....	27
USING THE SNAPSHOT BUTTON	27
3. USING THE COMPACT 6 HD SPEECH TEXT-TO-SPEECH AND OCR	28
4. CHARGING THE BATTERY	30
BATTERY WARNING	31
STANDBY	31
5. EASY AND ADVANCED MODE	32
6. FEATURES IN EASY MODE	34
ADJUSTING THE MAGNIFICATION	34
SELECTING A VIEWING MODE	34
7. FEATURES IN ADVANCED MODE	35
TIME AND DATE	35
MENU	36
<i>Voice (Compact 6 HD Speech only)</i>	38

Line On/Off	39
Battery status	39
Save	39
Open.....	40
Adjusting the brightness	41
SETTINGS	42
Color.....	43
View (Compact 6 HD Speech only).....	44
Miracast: connecting wirelessly to a monitor or TV	44
Bluetooth (Compact 6 HD Speech only).....	46
Line.....	48
Buttons	48
Theme	48
Columns (Compact 6 HD Speech only).....	48
Lights.....	49
Sound.....	49
Standby	49
Language (Compact 6 HD only).....	49
Restore.....	50
APPENDIX A: TECHNICAL INFORMATION.....	51
APPENDIX B: SAFETY.....	53
APPENDIX C: WARRANTY CONDITIONS.....	54

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Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on February 13, 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure the safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

1. Introduction

Congratulations on choosing the Compact 6 HD or Compact 6 HD Speech!

Optelec's unique pocket-sized 6-inch touch screen electronic video magnifier enables you to view and magnify printed text and images in high resolution HD quality in full color or high contrast colors. With its 6-inch display, two cameras, customizable touch screen interface and unique accessories, you can customize your Compact 6 HD and tailor it to your personal needs.

If you have questions or suggestions concerning the use of this product, please contact your distributor or Optelec's headquarters, using the contact information on the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your Compact 6 HD!

About this manual

At Optelec, we are constantly improving our products and their functionality. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on www.optelec.com at the Support section.

This manual will familiarize you with the features and basic operation of the Compact 6 HD and Compact 6 HD Speech. Please read this manual carefully before using your device. Unless otherwise indicated, all references to the Compact 6 HD will also apply for the Compact 6 HD

Speech. Functionality specifically applicable for the Compact 6 HD Speech are marked with the Speech logo.

Speech logo

What's in the box?

The Compact 6 HD packaging contains the following:

- The Compact 6 HD or Compact 6 HD Speech video magnifier and Reading Stand
- A protective pouch
- A wrist strap
- A USB power supply with USB-C cable
- 4 country-specific power plugs (EU, US, UK, and AU)
- A USB to 3.5mm audio plug converter
- A screen cleaning cloth
- This user manual

If any of these items are missing from your package, please contact your distributor.

2. Getting started

The Compact 6 HD is placed correctly in front of you when the screen is facing up, and the white snapshot button and orange on / off button are at the top corners of the screen.

Using the Compact 6 HD with the Reading Stand

The Compact 6 HD is delivered with a Reading Stand. When placed in its Reading Stand, the Compact 6 HD can be used with the Reading Stand closed or opened.

Compact 6 HD open


Compact 6 HD closed

In the **open** position, the screen will be set at an ergonomic angle for reading, and the magnifier camera will be activated. The open position enables you to read longer texts.


In the **closed** position, the overview camera is being used. Hold the unit in your hand and point the camera (located at the back of the unit) to the object or text you wish to view or read. The closed position is ideal for quickly scanning and viewing objects or documents.

To **insert** the Compact 6 HD into the Reading Stand, follow these steps:


1. Insert the Compact 6 HD at the bottom of the Reading Stand.
2. Press to lock the Compact 6 HD in the Reading Stand.
3. The Compact 6 HD is now locked in the Reading Stand.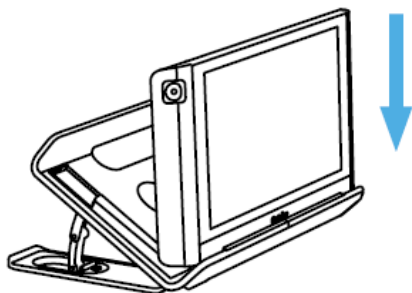

Step 1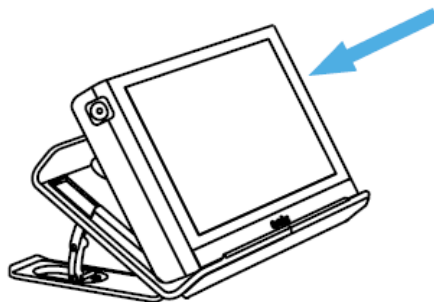


Step 2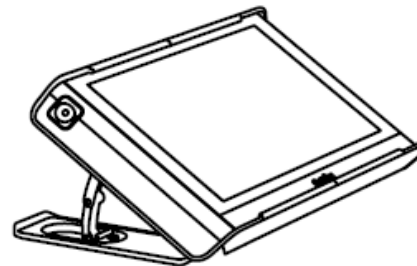

Step 3

To **remove** the Compact 6 HD from the Reading Stand, follow these steps:

1. Open the Reading Stand.
2. Place 2 fingers in the indents at the back.
3. Gently press to remove the Compact 6 HD from the stand.

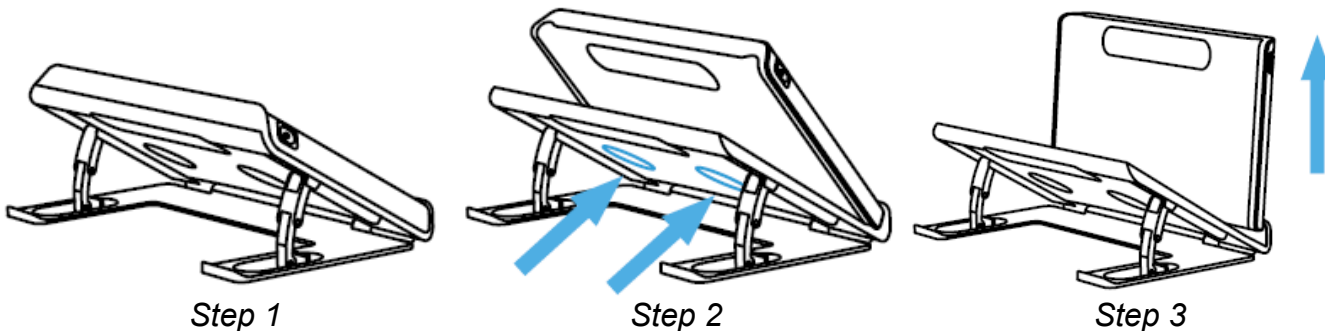To **open** the Reading Stand, follow these steps:

1. Place the Compact 6 HD in front of you so that the screen is facing up, and the white snapshot button and orange on / off button are at the top of the screen.
2. Open the feet of the stand, and the Compact 6 HD will switch on.

To **close** the Reading Stand, follow these steps:

1. Place the Compact 6 HD in front of you so that the screen is facing up, and the white snapshot button and orange on / off button are at the top of the screen.
2. Gently press the Compact 6 HD down to close the Reading Stand.

Cleaning the camera window

For an optimal quality image, it is recommended to ensure that the camera lens and object light window are kept clean. A dirty camera window will result in a fuzzy image and may cause the auto-focus camera to incorrectly focus. To clean the camera window, use the provided cleaning cloth or another soft lens cleaning cloth.

Note: Do not use water or cleaning agents to clean the camera window.

Buttons

The Compact 6 HD has the following buttons:

On / Off button

A short press of the orange button, located on the top left, will place the Compact 6 HD into sleep mode or resume the unit. A 3 second press of this button switches the unit off or back on again.

Snapshot button

The snapshot button located on the top right of the unit allows you to take a temporary picture of a distant text or image, enabling you to bring the image closer to your eyes. If you own a Compact 6 HD Speech, the Compact 6 HD will extract the text from the snapshot and read it aloud.

USB connector

The white USB connector is located on the right side at the bottom corner of the Compact 6 HD. The supplied USB cable connects to this USB port and power supply. Use only the power supply that comes with the Compact 6 HD to power and charge the unit. Use the USB to 3.5 mm audio converter cable to connect speakers or a headset.

Cameras

The cameras of the Compact 6 HD are located at the back of the unit.

Power on and off

To switch on the Compact 6 HD for the first time, the unit needs to be charged. Once charged, press the orange on / off switch located on the top left corner of the screen for three seconds. To switch off the Compact 6 HD, press the orange button for three seconds again. When switching on the Compact 6 HD, it will take approximately 25 seconds before an image displays on screen.


Sleep and resume

When the Compact 6 HD is switched on, a short press of the orange power button will place the unit into sleep mode. A short press of the orange power button again will resume the Compact 6 operation in the mode in which it was put to sleep. When resuming from sleep, the Compact 6 HD will take approximately 1 second before an image displays on screen.

When the Compact 6 HD is in sleep mode, the unit can also resume by opening the Compact 6 HD reading stand. Closing the reading stand will automatically put it back into sleep mode.

The magnifier and overview cameras

The Compact 6 HD is equipped with two cameras, a magnifier camera and an overview camera.

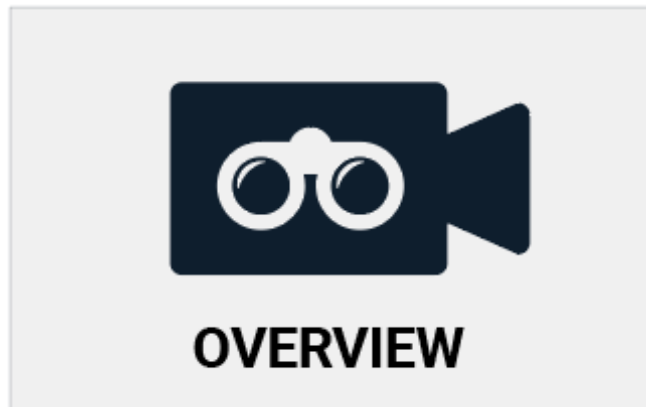When the stand is being opened, the Compact 6 HD will switch on and will automatically select the magnifier camera. When you switch on the Compact 6 HD with the stand closed, the overview camera will be activated. The magnifier camera is optimized for reading text in the reading stand while the overview camera is optimized to look at objects at a short distance. To switch between the cameras, tap the screen and tap the camera icon at the top of the screen.

Focus lock

The auto focus of the overview camera can be locked for writing or viewing objects at a fixed distance. To lock the focus, tap the screen for 2 seconds. To unlock the focus, tap the focus lock button at the top of the screen next to the camera switch icon.

Using the snapshot button

The snapshot button allows you to take a temporary picture of a text or an image.

1. Hold or place the Compact 6 HD over the text or image you wish to capture.
2. Press the snapshot button located on the top right side of the unit to take the snapshot.
3. Bringing the Compact 6 HD closer to your eyes will enable you to view the object or text closely.
4. Press the snapshot button again to and return to the live video mode. It is now possible to take a new snapshot.

To save or delete snapshots, please refer to chapter 7.

3. Using the Compact 6 HD Speech text-to-speech and OCR

If you own a Compact 6 HD Speech, taking a snapshot activates the Speech functionality. The Compact 6 HD Speech will extract the text from the image and will read the text aloud.

Activating the speech

To take a snapshot and activate the Speech:

- Position the text or part of the text in the camera view and press the snapshot button. For an optimal result, make sure there is enough light in the environment, prevent shadows and limit the text to only the part of the document you would like to read. If the text in view gets too small, the recognition will lose accuracy and might fail.
- A progress bar will be shown. The Compact 6 HD Speech will start reading after the recognition process has been finished.
- Use the Play/Pause button to start or stop reading
- Tap and hold (long press) on the text to start reading at the word or section you point at.
- Tap the “Mode” button to display the text in high contrast colors. The text will be formatted in a column to fit the width of the screen. In the high contrast view, pictures will not be shown. Press the “Mode” button to switch between the various high contrast options and full color mode. In the full color mode pictures will be shown as well as the original layout of the text.

- If no text is found, a “No text found” message will appear on pressing the “Play” button.
- To change the language and voice, please refer to chapter 5.

Adjusting the volume and reading speed

To adjust the volume or reading speed, activate the button bar with a single tap. Tap the Audio button to display the “Volume” and “Speed” sliders. The “Volume” and “Speed” sliders are only present in the snapshot view.

Adjusting language and voice

To adjust the language or voice in Easy Mode, press and hold the “Audio” button to go to the Voice menu. To adjust the language or voice in Advanced Mode, press the “Menu” button and then press the “Voice” menu. Please refer to Chapter Chapter 7, Voice for more information on how to navigate the “Voice” menu.

Table mode

The regular reading mode reads columns, ideal for newspapers and magazines. But, you might occasionally want to read a menu, receipt, table or other text that has a table-like layout. By pressing and holding the Snapshot button, the Compact 6 HD Speech will read all captured text per line from left to right first, and then move on to the next line. When you exit the snapshot and take another snapshot, you will automatically switch back to the regular reading mode.

4. Charging the battery

The Compact 6 HD is delivered with a power supply and four country-specific power plugs in EU, US, UK, and AU types. Select the power plug that corresponds to your country and plug it into the power supply.

To charge the Compact 6 HD, connect the supplied USB cable to the power supply and the white USB connector located on the right side of the unit. Plug the other end of the power supply into a power outlet.

Battery full

Battery charging

Battery almost empty

Charging the Compact 6 HD takes approximately 2 hours. A fully charged Compact 6 HD can operate for an average time of 3 hours, depending on the power settings. For safety reasons, please only use the power supply and USB cable that was delivered with the Compact 6 HD to charge the unit's battery.

Battery warning

The Compact 6 HD will briefly show an empty battery warning icon to indicate that the battery needs to be charged. When this occurs, make sure to charge your unit. When the battery is empty, the unit will automatically switch off.

If the Compact 6 HD does not switch on, please make sure it is charged.

Standby

The Compact 6 HD can automatically go into standby mode if no button is pressed or no movement is detected. Pressing the orange on / off button will switch it back on. The standby time can be changed or disabled in the Compact 6 HD menu. Please refer to chapter 7 on how to change these settings.

5. Easy and Advanced mode


Upon first using the unit, it will start up in Easy mode. To switch between Easy and Advanced mode, press and hold the Mode button and tap Easy or Advanced.

In Easy mode, you have access to the main features:


- Adjusting the magnification
- Selecting a viewing mode
- Taking a snapshot
- If you own a Compact 6 HD Speech,
 - The text in the snapshot will be read out aloud.
 - The language, voice, reading speed and volume can be adjusted.
- Switching between the Magnifier and Overview camera

In Advanced mode, you have the same functions as in Easy mode, plus some additional features:

- Time and date
- Access to the menu with the options to:
 - Save and open snapshots
 - Changing the language and reading voice
 - Changing color modes
 - Enabling the Reading Line
 - Bluetooth audio and Miracast
 - Other advanced settings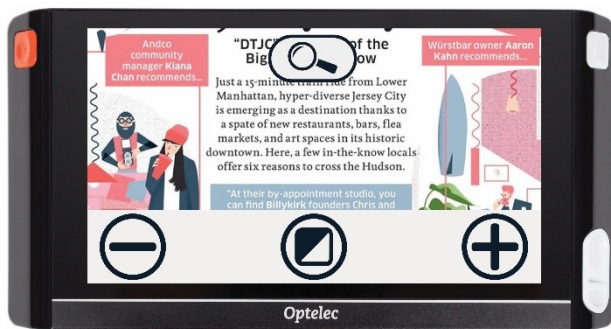

Easy mode

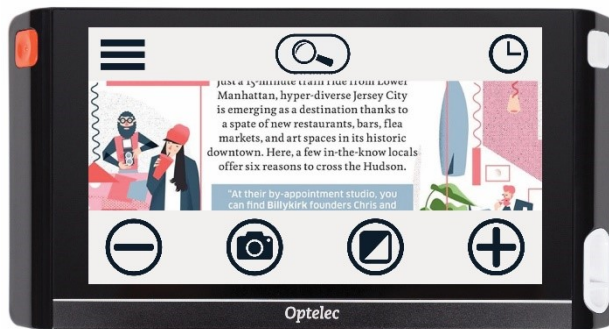Advanced mode

6. Features in Easy mode

Adjusting the magnification

To change the magnification, simply tap the screen. The button bar will be shown at the bottom of the screen or at the right side of the screen, depending on the chosen button bar position. The button bar position can be selected in the menu. See chapter 7 for more information on how to change the button bar position. In the button bar tap the **+** button to increase or the **-** button to decrease the magnification. Alternatively, use the pinch-to-zoom gesture to change the magnification. Tap the screen to hide the button bar. The button bar will automatically hide when the screen is not touched for 10 seconds.

Selecting a viewing mode


Tapping the Mode button in the middle of the button bar allows you to select one of the following viewing modes:

- **Photo mode:** Viewing full color texts and photos
- **Negative mode:** Reading in high contrast, white on black
- **Positive mode:** Reading in high contrast, black on white
- High contrast **color combination 1**
- High contrast **color combination 2**

7. Features in Advanced mode

Time and date

To open the clock, tap the screen and tap on the “Clock” icon at the right edge of the screen. Or, swipe in from the right side of the screen.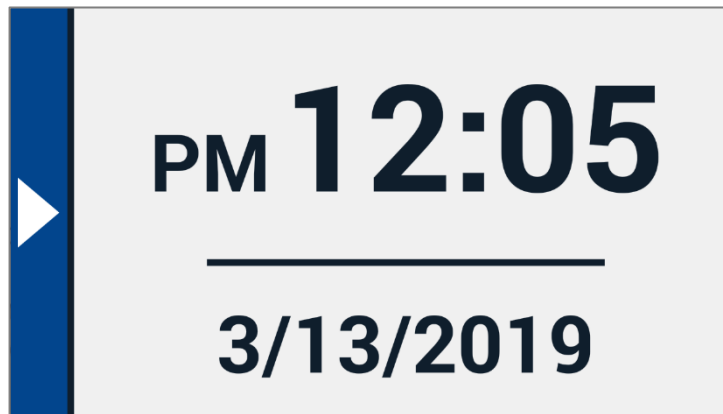


Use the “Back bar” to exit the time and date screen.

To set the time and date, tap the time or date. To change a value, tap on the field you would like to change and tap on the up or down symbol to change the value. Tap on the “Back bar” to exit the set time and date screen.

Menu

Opening and closing the menu

To open the menu, tap on the screen and tap on the “Menu” icon at the left side of the screen. Or, swipe in from the left side of the screen. This will open the main menu. To close the menu, tap on the “Back bar”.

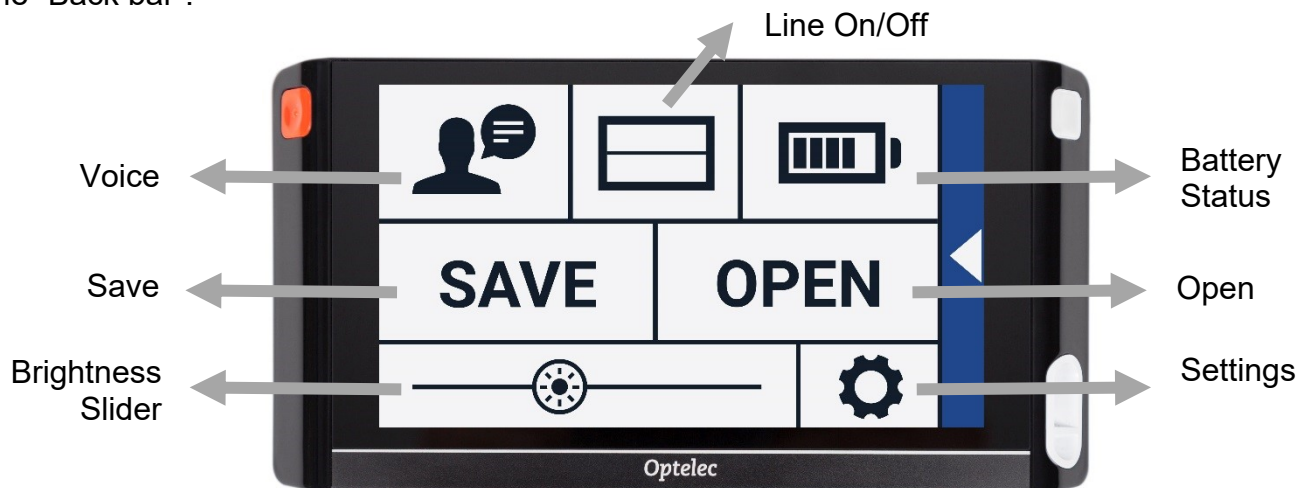The exact items in the menu will vary, depending on if you own a Compact 6 HD or 6 HD Speech. To activate a button, tap the button once. Activating a button will open a new menu screen or list.

To go back to the main menu, tap on the “Back bar”. To adjust the slider, tap and hold the slider button while moving it to the right or left.

The main menu contains the following options:

- Voice (Compact 6 HD Speech only)
- Line On/Off
- Battery status
- Save
- Open
- Brightness slider
- Settings

Voice (Compact 6 HD Speech only)

To select a voice, language and menu language, or to add or remove a voice or language, open the menu and tap the “Voice” button.

Selecting a voice, language and menu language

This menu item lets you select your reading voice and language. To select another reading voice, tap the voice icon in the top left corner of the main menu to open the list with preselected voices. Each voice represents a language. Tap on a voice to select the voice. If adjusting the reading voice results in a change of language, the menus will automatically change to that language.

Adding or removing a voice or language

To add or remove a voice or language, tap the voice icon in the top left corner of the main menu. In the list with preselected voices, tap on the “Add/Remove” button. To add or remove a reading voice, select the language from the list by tapping on the language. This will display a list with available voices for the selected language. To enable a reading voice, select it from the list by tapping on it. A checkmark in front of the voice name will indicate if it is enabled. When the voice is enabled, it will be available under the “Voice” button in the main menu. By deselecting a voice, the voice will be removed from the available reading voices. The languages of which a reading voice is selected, will appear at the top of the list with languages, and are marked with a checkmark.

Adjusting language and voice in Easy Mode

In Easy mode, you cannot access the Menu. To adjust the language or voice in Easy Mode, press and hold the “Audio” button to go directly to the Voice menu and follow the steps described above.

Line On/Off

The Compact 6 HD can display a reading line to help you position the Compact 6 HD straight on the document. To enable the reading line, tap the “Line” button. Tap this button again to hide the reading line. The position of the reading line can be set in the settings menu.

Battery status

The battery status is shown in the top right corner of the screen. If you tap the battery icon, a new screen will be opened showing a magnified battery icon and battery status, the “About” button and the “Back arrow” to return to the main menu. The lightning symbol indicates that the battery is charging.

About

Tap on the “About” button in the battery screen to see the application version.

Save


To save a snapshot, open the menu and tap the “Save” button.

- Make a snapshot using the snapshot button
- Open the menu
- Press the save button to save the snapshot
- The menu will close and the snapshot will be shown


If there is no snapshot to save, or if the snapshot has already been saved, the “Save” button will be hidden.

Open

To open a snapshot, open the menu and tap on the “Open” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. If there are more snapshots than fit on the screen, swipe up and down to scroll through the list. Tap on the snapshot thumbnail to open it.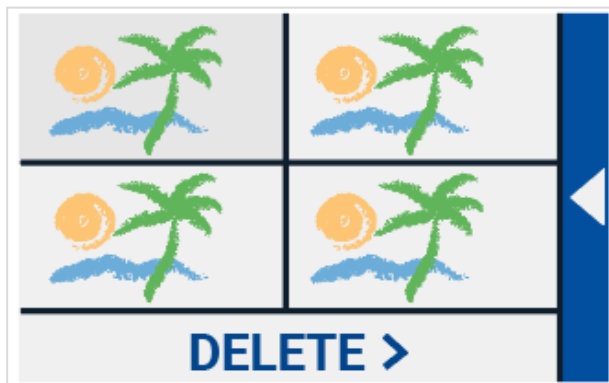

Open to view saved snapshots

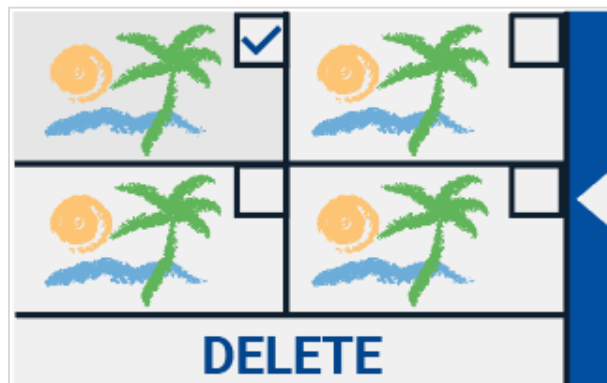Delete snapshots

Delete

To delete a snapshot, open the menu and tap on the “Open” button. In the open screen, tap on the “Delete” button. A list of thumbnails of the saved snapshots will be shown, showing the most recently saved snapshot first. Each snapshot has a checkbox at the top right corner. To select a snapshot, check the checkbox. Tap the “Delete” button to delete the selected snapshots. To delete all snapshots, tap the “Delete” button without selecting a snapshot. You will be prompted to confirm to delete all snapshots. To delete all snapshots, select “Yes”. To cancel deleting all snapshots, select “No”. Both choices will return you to the list of snapshots. Tap the “Back bar” to return to the “Open” screen. Tap the “Back bar” again to return to the main menu.


Adjusting the brightness

Use the slider with the sun icon to increase or decrease the brightness of the screen.

Settings

Tap on the “Settings” button to enter the Settings menu. The “Settings” button is the button with the gear icon at the bottom right. The Settings menu contains the following buttons:

- Color
- View (Compact 6 HD Speech only)
- Miracast
- Bluetooth (Compact 6 HD Speech only)
- Line
- Buttons
- Theme
- Columns (Compact 6 HD Speech only)
- Lights
- Sound
- Standby
- Language (Compact 6 HD only)
- Restore

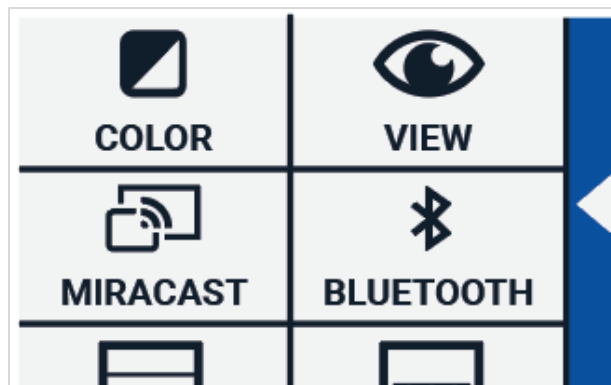Color

In the settings menu, tap the “Color” button to select the preferred high contrast colors. The Compact 6 HD supports four high contrast modes. For each of these four modes, the foreground and background colors can be selected. Three of the high contrast modes can be disabled. The first high contrast mode on the top left is required.

To change the foreground and background colors, tap the up or down arrow next to the mode you want to change. You will cycle through each of the available modes and can stop on the one you prefer. Select the “X” symbol to disable a high contrast mode. Tap on the “Back bar” to return to the settings menu.

View (Compact 6 HD Speech only)

In the settings menu, tap the “View” button to configure how you want to display captured text when using Text-to-Speech.

To configure how all captured text is shown, press the Mode button to cycle through the different viewing modes: Original, Line and Column.



To configure how the word that is read aloud is shown, press the Highlight button to cycle through the different highlight modes: Rectangle and Underline.

When using Text-to-Speech, tap the Mode button in the button bar to view captured text in the viewing mode you just selected and cycle through your configured high contrast colors.

Miracast: connecting wirelessly to a monitor or TV

Miracast is a standard for a wireless connection to a monitor. It can be described as "HDMI over Wi-Fi", replacing the cable from the Compact 6 HD to the monitor or TV. To be able to use the Miracast function of the Compact 6 HD, your monitor or TV needs to support Miracast. Please refer to your monitor or TV manual and documentation to verify if your device supports the Miracast feature.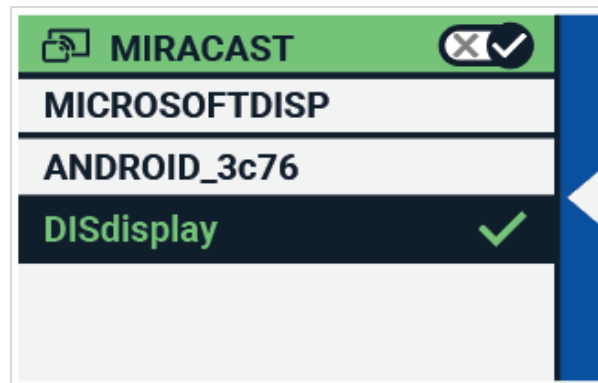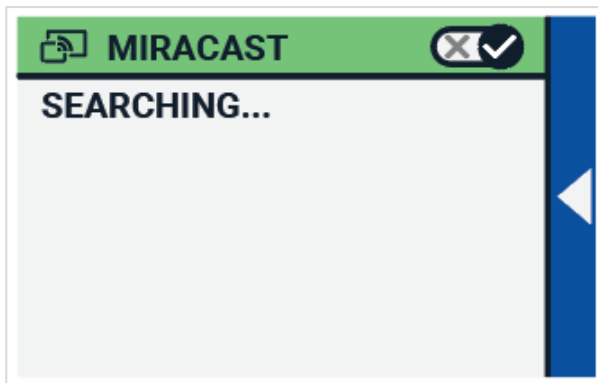


To connect your Compact 6 HD to a monitor or TV via Miracast, tap the “Miracast” button and Switch Miracast on. A list with available devices will be displayed. Click on a device to connect. It could take up to a minute to establish the connection. Tap on the device again to disconnect. When a Miracast connection is established, both the audio and video will be streamed to the TV or monitor. To guarantee a good connection, the Compact 6 HD needs to be within one meter or three feet of the monitor or TV. Depending on the connection, a short delay of the video and / or audio could be noticed.

Bluetooth (Compact 6 HD Speech only)

To connect a Bluetooth speaker or headphone, tap the “Bluetooth” button and switch Bluetooth on.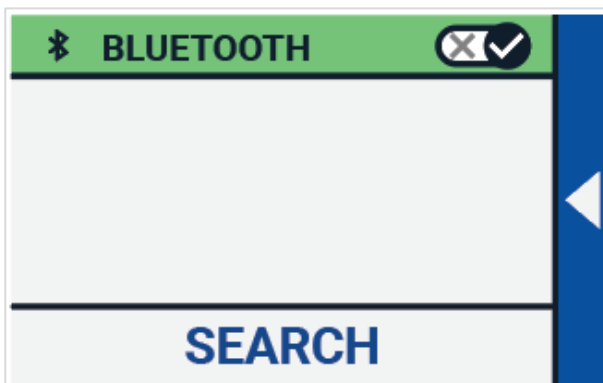


A list with paired devices is being displayed.

If no speaker or headphone is being paired, tap the “Search” button to search for available devices. Before starting the Bluetooth search, make sure the speaker or headphone is switched on and is set to discoverable. Please refer to the speaker’s or headphone’s manual on how to switch the device to Bluetooth discoverable mode. To pair the speaker or headphone, tap on the device name

and if needed enter the pin code. Refer to the speaker's or headphone's manual for the pin code of the device.

When the device is paired, to connect the speaker or headphone, tap the device. You will be asked to connect or forget the device. Tap the "Connect" button to make the connection.

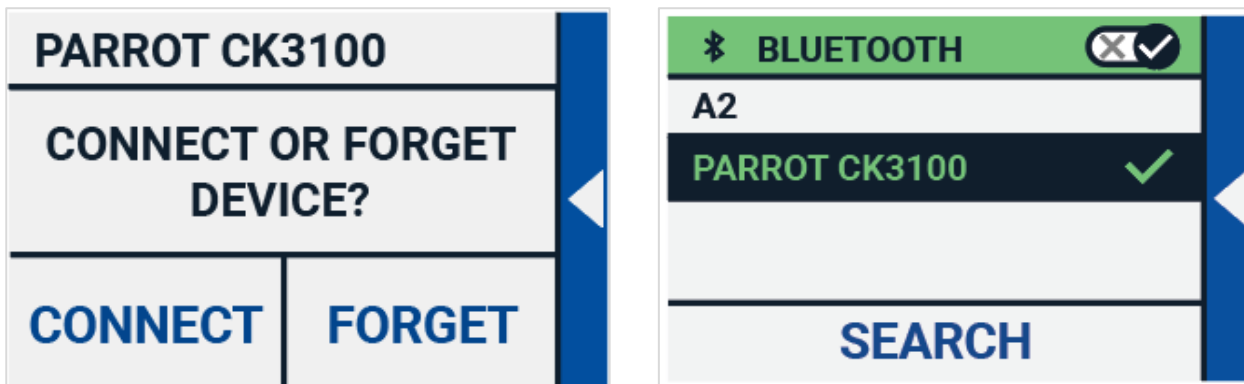To indicate that a device is connected, the connected device will have a checkmark and will be displayed in blue.

To unpair / forget a device, tap the "Forget" button. The device will be removed from the list with paired devices.

Note: Only Bluetooth version 4.0 or higher headsets and speakers are supported.

Line

This menu item enables you to change the reading line position to one of its six positions; horizontal top, horizontal center, horizontal bottom, vertical left, vertical center, vertical right. By default, the reading line is horizontally set to the centre of the screen. By changing the reading line position, the reading line will automatically be enabled. The reading line can be disabled in the main menu.

Buttons

This menu item enables you to change the button bar position to one of two positions: bottom or side. Depending on your preference, you could change the position of the button bar to the bottom or side of the screen. The default setting is “Bottom”.

Please note that the bar position could be different, depending on the accessory in use.

Theme

This menu item enables you to select a light or dark theme for the buttons and menus.

Columns (Compact 6 HD Speech only)

This menu item allows you to choose if you want the Compact 6 HD Speech to only read full columns, or incomplete columns as well. By default, Read full columns only is selected. Please note that if you select to read incomplete columns, not all characters and words are fully captured, which may result in incoherent text.

Lights

This menu item allows you to enable or disable the lights in Magnifier mode and in Overview mode. The Magnifier lights are located to the left and right of the magnifier camera. By default, the reading lights are enabled for an optimal image quality.

The Overview light are located next to the overview camera. By default, the overview light is disabled.

Sound

This menu item allows you to change the volume, mute the audio feedback and enable or disable the vibration feedback. By default, sound and vibration are enabled.

Standby

The standby time sets the time after which the Compact 6 HD goes into standby mode to conserve power. Pressing any button will take the unit out of standby mode. By default, the standby time is set to 5 minutes after no button is pressed or no movement of the Compact 6 HD is detected.

Language (Compact 6 HD only)

This menu item allows you to choose the language of the Compact 6 HD. Tap to select your preferred menu language.

Restore

Tap this menu item if you wish to restore the default settings. You will be prompted if you are sure to restore the defaults. To restore the defaults select “Yes”. To cancel restoring the defaults, select “No”.

Appendix A: Technical information

Magnification	<0.5 – >21 times
Viewing modes	Full color photo mode High contrast, white on black High contrast, black on white High contrast color combination 1 High contrast color combination 2
Focus	Auto-focus
High contrast ratio	700:1
High brightness screen	350 cd/m ²
Screen	5.94" TFT
Screen resolution	1280 x 720
Dimensions	183 x 95 x 14 mm / 7.2 x 3.7 x 0.55 in
Weight Compact 6 HD	270 grams / 9.5 ounces
Weight Reading Stand	110 grams / 3.0 ounces
Battery	average 3 hours continuous use average 2 hours charging time
Li-poly battery	rechargeable, 3.7V 2600mAH
Power supply type	Intertek, MX15Z-0502500VU
Input power rating – Adaptor	100-240V, 50-60Hz, 0.4A
Output power rating – Unit	DC5V-2.5A

Operating conditions

Temperature	+10°C to 35°C / 50°F to 95°F
Humidity	< 70%, no condensation
Altitudes	up to 3000 m (9842 feet)
Pressure	700 – 1060 hPa

Storage & transportation conditions

Temperature	+10°C to 40°C / 50°F to 104°F
Humidity	< 95%, no condensation
Altitudes	up to 12192 m (40000 feet)
Pressure	186 – 1060 hPa

Appendix B: Safety

Intended Use:

The Compact 6 HD is designed that it can be used in a hospital environment for general purpose such as reviewing medical records. The Compact 6 HD cannot be used for surgical procedures in combination with life supporting systems.

Warning: Critical diagnostic decisions should not be based on images displayed by this device.

- Do not expose the Compact 6 HD to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Compact 6 HD. Contact an Optelec authorized service organization in your area.
- To avoid risk of electrical damage, keep your Compact 6 HD away from fluids and chemicals.
- Handle the Compact 6 HD with care. Rough handling will damage the internal components.
- Do not use the Compact 6 HD near inadequately shielded medical devices.
- Do not try to open the battery compartment or try to remove the battery.
- Contact your dealer to service this equipment if necessary. **Do not disassemble your unit** as it will void the warranty.
- Always unplug and switch off the unit before cleaning. Use a dry soft cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.

Use of the Compact 6 HD other than described in this manual will exclude it from warranty conditions.

Appendix C: Warranty Conditions

Optelec warrants the Compact 6 HD, effective from the date of delivery, to be free of any defects in material and workmanship.

The warranty is not transferable and does not apply to groups and multi-users. The Compact 6 HD was designed for the individual purchaser to be used at home or on the go. Optelec reserves the right to repair or replace any purchased Compact 6 HD with a similar or improved product.

In no event shall Optelec or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of the Compact 6 HD. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after the warranty period, please contact your Optelec distributor.

Optelec is not responsible for any uses of this device other than those described in this manual. Use of the Compact 6 HD other than described in this manual will exclude it from warranty conditions.

Grounding reliability can only be achieved when the Compact 6 HD power supply is connected to power outlet which has an equal medical certification.

Warning – Do not modify this equipment without authorization of the manufacturer.

Warning – To avoid risk of electric shock, this equipment must only be connected to a grounded power outlet.

Caution: The supplied adapter is part of the Compact 6 HD product.

Caution, consult ACCOMPANYING DOCUMENTS.

Follow the operating instructions or consult instructions for use.

STAND-BY.

Direct current.

RoHS Compliance

This product is in compliance with Directive 2011/65/EU of the European Parliament and of the Council of 03 January 2013, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

European Union Notice

Products with CE marking comply with the EMC Directive (2014/30/EU) and the Low Voltage Directive (2014/35/EU) issued by the Commission of the European Community.

WARNING: The use of accessories, transducers and cables, with the exception of transducers and cables sold by the manufacturer of the Compact 6 HD as replacement parts for components, may result in increased emissions or decreased immunity of the Compact 6 HD.

This device bears the CE and FCC approval mark

Federal Communications Commission (FCC) Statement:

This Equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

WARNING: You are cautioned that changes or modifications not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received; including interference that may cause undesired operation.

WARNING: Risk of explosion if the battery is replaced by an incorrect type. Please dispose of used batteries according to the disposal and recycling regulations that are applicable in your country or region.

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All rights reserved
U.S.A.
Optelec GmbH