Optelec

Compact Touch HD

User Manual

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Optelec

The Compact Touch HD will be helpful for low vision and blind people in reading.

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 13th February 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

RoHS Compliance

This product is in compliance with Directive 2002/95/EC of the European Parliament and of the Council of 27 January 2003, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

European Union Notice

Products with CE marking comply with the R&TTE Directive (99/5/EC), the EMC Directive (89/336/EEC), and the Low Voltage Directive (73/23/EEC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following international standards:

IEC 60950-1 2004;

IEC 55022

IEC 55024 (IEC61000-4-2 excluded)

FCC

FCC part 15 Class B

Compact Touch HD

User Manual

Version 1.2

© 2013 Optelec, the Netherlands

All rights reserved

Optelec

P.O. Box 399

2990 AJ Barendrecht

The Netherlands

Phone: +31 (0)88 678 34 44

E-mail: info@optelec.nl

Internet: www.optelec.com

Contents


1.	Introduction.....	6
2.	General description Compact Touch HD.....	6
3.	Intended use	7
4.	Safety and Maintenance.....	8
5.	Installing the Compact Touch HD	9
6.	Instructions for charging	10
7.	Battery handle maintenance.....	11
8.	Folding handle	12
9.	Focusing the Compact Touch HD	12
10.	Far View function.....	13
11.	Switching the Compact Touch HD on or off.....	13
12.	Switching the LED on/off	13
13.	Display modes	14
14.	Zooming	15
15.	Freeze function	16
16.	Magnetic lines.....	17
17.	Compact Touch HD menu.....	18
	Sound, brightness and reading lines	18
	Sound icon	19
	Brightness icon.....	19

Reading lines	20
18. Display modes and battery	21
Battery icon	22
Display mode icon	23
19. Picture output	24
20. Writing with the Compact Touch HD	25
21. Troubleshooting	26
22. Warnings	28
23. Discarding the Compact Touch HD or battery	28
24. Warranty regulations	28
25. Regulations	29
Appendix A: Safety	30
Appendix B: Specifications	31
Optelec offices worldwide	153

1. Introduction

Congratulations with the purchase of your Compact Touch HD! In this manual you will find a detailed description of all the functions of the Compact Touch HD. Read this manual carefully before you start using the Compact Touch HD. If you still have any questions after reading this manual, please contact your supplier.

2. General description Compact Touch HD

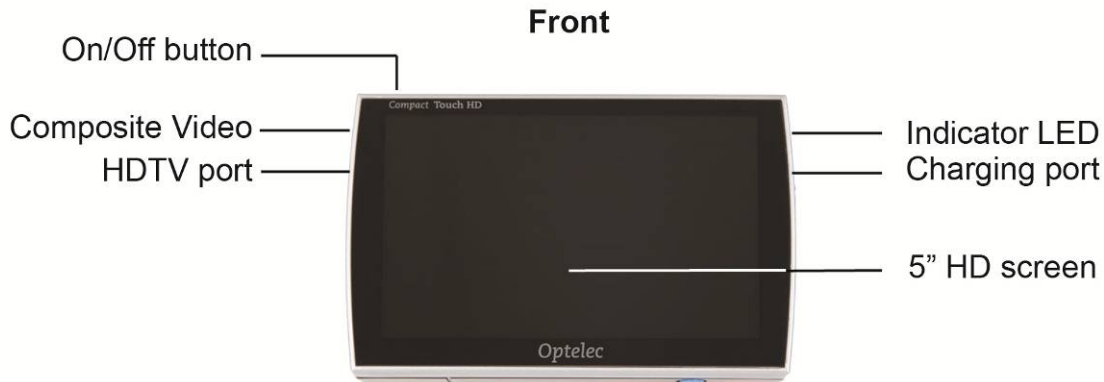Back

3. Intended use

This device is solely used for displaying enlarged representations of text, photographic material and objects. When the Compact Touch HD is used for any purpose other than the intended use, the CE regulation will not apply.

Caution: do not, under any circumstances, perform maintenance or repairs on the device; always ask a manufacturer approved technician to do this. Warranty will be voided if the Compact Touch HD is attempted to be repaired by anyone other than a technician approved by the manufacturer.

4. Safety and Maintenance

Before use, be sure to comply to with the following safety and maintenance instructions:

- Protect the device and its batteries from moisture, rain, sand or extreme temperatures like radiators or direct sunlight.
- Do not use cleaning products that contain alcohol, methylated spirits, ammonia or abrasives. These can damage the device or its touch screen.
- Only use accessories that the manufacturer supplies you with, or approves of.
- Never throw batteries into a fire. Batteries and chargers that are defect are considered chemical waste. They therefore need to be disposed of at a refuse/recycling centre of your choice.
- Note that the use of a screen protector might influence the functioning of the touch screen.

5. Installing the Compact Touch HD

Connect the battery handle of the Compact Touch HD by sliding it onto the connector until you hear it clicking into place. This battery handle also functions as the Compact Touch HD's handle.

To replace the battery handle push the 2 blue battery release buttons on the side of the battery handle, and slide on the replacement battery handle.

Charge the battery handle completely before using the Compact Touch HD. Please read the charging instructions carefully.

6. Instructions for charging

Put the adaptor in the wall socket and connect its plug to the charging adaptor port of the Compact Touch HD.

LED burning constantly	batteries are being charged
LED off	batteries are fully charged
LED blinking	error / batteries are defect

Charge the batteries in a well ventilated environment. Remove the Compact Touch HD from its carrying case when charging. Don't place it in direct sunlight or near a radiator whilst charging.

When the battery is empty, the Compact Touch HD will switch itself off. You will hear a high pitched tone that fades away.

7. Battery handle maintenance

- Remove the battery handle from the Compact Touch HD if you do not intend to use it for an extended period of time. This will improve the longevity of the battery handle.
- Batteries are sensitive to overcharging. It is recommended to only charge the battery handle when it is completely drained.
- Chargeable Lithium-Ion batteries have an auto discharge that strongly depends on the temperature of the environment (e.g. never leave the battery handle in a car that is parked in direct sunlight).
- New Lithium-Ion batteries must be charged several times before reaching their nominal capacity.
- Charge Lithium-Ion batteries at least once every 6 months.
- The average lifespan of the battery handle is usually 300 charging cycles, but this depends on the use. Total capacity will slowly decrease after multiple charging cycles.

8. Folding handle

The Compact Touch HD comes with a replaceable foldable handle with a built-in battery. This handle can be used in several positions.

When the handle is extended, you can use the Compact Touch HD as a magnifier. The closer you get to the object, the bigger the magnification will be. You can also hold the Compact Touch HD up to see distant objects as the Compact Touch HD can focus from 5cm to infinity.

It is also possible to use the Compact Touch HD with the handle folded in. You can easily set the device on the table and it will automatically stand at an angle. The magnification will then be about 8x. This is easier when you want some stability during the reading of texts.

9. Focusing the Compact Touch HD

The Compact Touch HD uses a push-to-focus method to focus the image on the screen. Simply touch the screen on the section where you want the Compact Touch HD to focus; the camera will focus on that area.

10. Far View function

The camera of the Compact Touch HD can focus on objects from 5 cm. distance to infinity. Just hold the device up to view the object. Remember to touch the screen in order to focus the camera on the object you are viewing.

11. Switching the Compact Touch HD on or off

To switch the Compact Touch HD on, press the ON/OFF button once. You will hear a scaled sound increasing in pitch.

To switch the Compact Touch HD off, press the ON/OFF button for 3 seconds. You will hear a scaled sound decreasing in pitch.

When you have the Compact Touch HD switched on, and it doesn't detect any movement on the screen, it will switch itself off after 3 minutes. A two-pitched signal will be heard.

12. Switching the LED on/off

By touching the centre of the screen for 1 second you can switch off the LED at the back of the Compact Touch HD. You will hear a beeping sound indicating that the LED is switched off.

Touch the centre of the screen again for 1 second to turn the LED back on. Another beeping sound will be heard.

13. Display modes

By swiping from the top to the bottom of the screen, or vice versa you can cycle through multiple colour modes.

or

14. Zooming

To alter the magnification you slide your finger up/down on the left or right hand side of the touch screen.

Zooming icon

To zoom in slide your finger upwards on the zooming icon, to zoom out, slide downwards on the zooming icon. The magnification range of the Compact Touch HD ranges from 2x to a 20x magnification.

15. Freeze function

By shortly pressing the ON/OFF button you can freeze the image on the Compact Touch HD's display. Press the button again to revert to the regular viewing mode.

When the image is frozen, the lighting will be dimmed to save battery capacity.

When the image is frozen, you can use 1 finger to pan through the frozen image. All functions of the Compact Touch HD can still be used while the image is frozen.

16. Magnetic lines

On the bottom side of the handle, there is a magnetic area that can be used with the magnetic sheet.


Slide the sheet underneath the text you want to read. Now place the Compact Touch HD on the text and slide it across the magnetic sheet. The ridges in the magnetic sheet will guide the Compact Touch HD along the lines; making it easier to keep the Compact Touch HD straight while reading.

17. Compact Touch HD menu

Sound, brightness and reading lines

Use 1 finger to touch the bottom section of the screen for 2 seconds to generate the sound menu, brightness menu and the reading line option in the bottom half of the screen. These settings will be saved when the Compact Touch HD is switched off.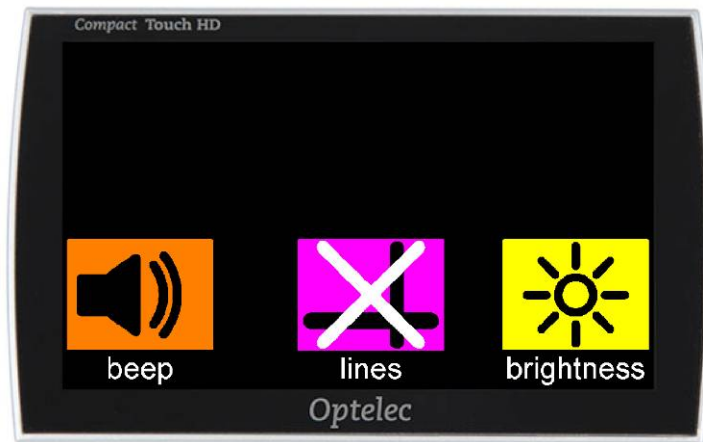

Sound icon

With this icon you can switch the sound on or off.

sound on

sound off

Brightness icon

With this icon you can set the brightness of the screen. There are 5 different intensity settings.




least brightness

most brightness


Reading lines

By tapping this icon you can generate a horizontal or vertical reading line on the screen.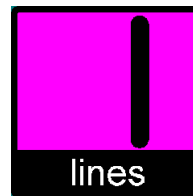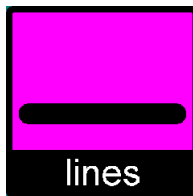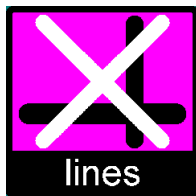


18. Display modes and battery

Use 1 finger to touch the top section of the screen for 2 seconds to generate the battery and display mode menu in the top half of the screen.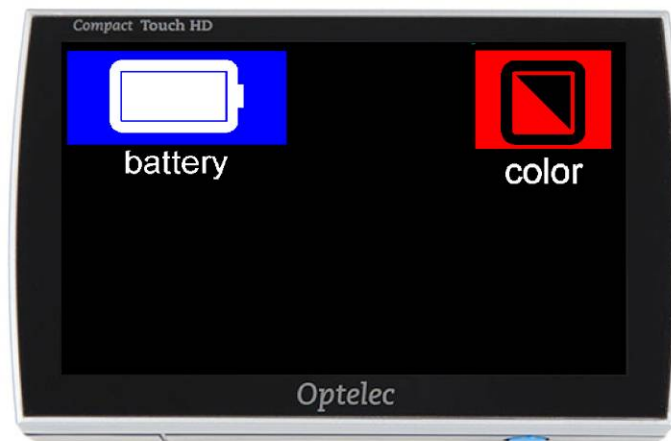

Battery icon

This icon indicates the remaining battery life. There are 5 different icons to indicate this.

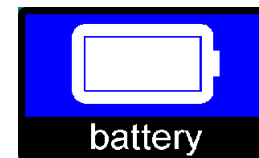battery empty

battery full


When the battery is charging, the menu will display the charging icon.

When the adaptor is connected, but the battery is not connected, this icon will appear.


Display mode icon

By touching the display mode icon, an overview of all available display modes will appear. You can easily select which display mode you want to use.

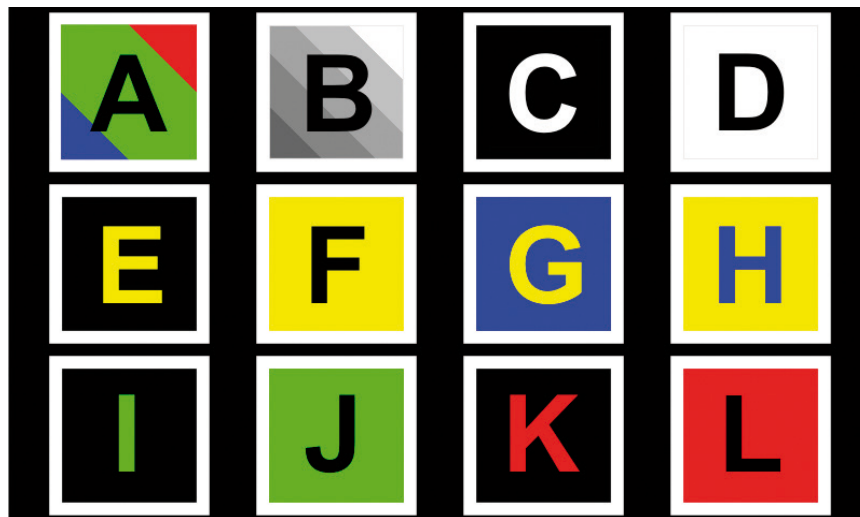19. Picture output

The Compact Touch HD can output the picture to an external display. Do this simply by connecting the appropriate cable to the HDTV port or the Composite Video port. Be sure to connect the Compact Touch HD to an earthed TV for the best results.

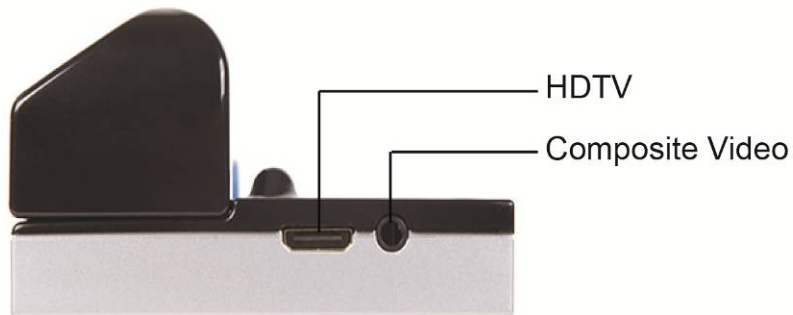20. Writing with the Compact Touch HD

Fold the handle inwards and place it on the paper you want to sign.

21. Troubleshooting

If the following table does not offer a solution to your problem, please contact your distributor.

If there is no picture or a dark or blurry image:

- Switch the Compact Touch HD on.
 - Image is frozen, shortly press the ON/OFF button once.
 - Switch the viewing mode to colour.
-

If the Compact Touch HD cannot be switched on:

- Press the ON/OFF button to switch the Compact Touch HD on.
 - Charge the battery handle.
 - Reinsert or replace the battery handle.
-

If the Compact Touch HD switches itself off within minutes, sounding a dual-pitched tone:

- The Compact Touch HD switches itself off if it does not detect movement on the screen for 3 minutes.

If the Compact Touch HD switches itself off within minutes with a single high pitched tone that fades away:

- Charge the battery handle
- Replace the battery handle with a new battery handle

22. Warnings

This icon indicates that users with pacemakers should take care with the magnetic area of the handle on the Compact Touch HD as this might, depending on your pacemaker, interfere with the pacemaker. Please contact your surgeon to inform yourself on the effects of magnets with your pacemaker.

23. Discarding the Compact Touch HD or battery

This symbol of the crossed out wheelie bin indicates that the product must not be disposed of with your other household waste. Check local regulations for proper disposal of electronic products.

24. Warranty regulations

This device is subject to the warranty regulations of your supplier. For further information regarding these regulations, please contact your supplier.

25. Regulations

Products with CE marking comply with EMC Directive (2004/108/EC), and the Low Voltage Directive (2006/95/EC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following standards:

EMC: EN 55022; EN 55024

FCC: part 15 Class B

Safety: EN 60950

Appendix A: Safety

- Handle the Compact Touch HD with care. Rough handling will damage internal components.
- Contact your Optelec distributor to service this equipment if necessary.
- **Do not open or remove any parts** of the Compact Touch HD as it will void the warranty.
- Do not try to remove the battery.
- Do not expose the Compact Touch HD to excessive heat or direct sunlight to avoid the risk of fire.
- Do not use the Compact Touch HD near inadequately shielded medical devices.
- To avoid risk of electrical damage, keep your Compact Touch HD away from fluids and chemicals.
- Always unplug the Compact Touch HD before cleaning. Use a soft cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.

Use of the Compact Touch HD other than described in this manual will exclude it from warranty conditions.

Appendix B: Specifications

Main specification

Measurements	131 × 75 × 32 mm (l × w × h)
Weight	300 grams (including battery)

Display

TFT diagonal	5.0"
Other specifications	High Brightness Touch technology

Camera

Minimum magnification	2×
Maximum magnification	20×
Focus	5cm - infinity

Power supply

Battery system	Interchangeable (inside battery handle) Lithium-Ion 4000mAh
Adaptor	5VDC/1.8A

Conditions of use

Temperature	+5°C till +35°C / 50°F till 95°F
Air humidity	<70% (no condensation)

Time of use and charging time

Time of use	Up to 3 hours
Charging time	approx. 5 hours

Package contents

Compact Touch HD
Battery handle (4000mAh)
Charging adapter
Travel pouch
Magnetic sheet
User manual
HDTV cable
SDTV cable
Cleaning cloth

This device bears the CE and FCC approval mark.

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Optelec Limited
Optelec GmbH